



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (03-2014)

Serial Number <b>80-005948</b>	Instrument Location <b>Walsh County Interview Room</b>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): None
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Paul An Inspector</i>	Date <b>6/24/16</b> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <b>27 June 16</b>

Intoxilyzer Test Record and Checklist  
 NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
 North Dakota Model 8000      SN 80-005948  
 Location = GRAF      8164.13.00 06/09  
 06/24/2016      14:55

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=|  
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=|  
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:      Enabled  
 Start Test Sequence:      DABACABA  
 Display Prelim Rslt?      Yes  
 Display Third Digit?      Yes  
 Inhib Printer(Y/N)?      No  
 Display Volume?      No  
 Disable On Memfull?      Yes  
 # of Print Copies?      1  
 Select Std (D/W/I)?      Dry  
 Standard Value?      0.080  
 Standard Lot #?      01316080A2  
 Standard Cyl #?      27  
 Standard Expiration?      02/05/2018  
 Oper No?      130890

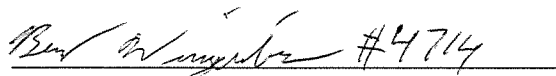
Flow Cal. Date:      06/20/2013  
 Slope      701  
 Intercept      -716431

IR Calibration Date:      04/02/2015

	3um	9um
--	-----	-----

	-----	
0th Coef(*100):	-25939	-20035
1st Coef(*100):	272687	134661
2nd Coef(*100):	3113	1576
H2O adj(mg/l*10k):	531	397

\*\*\*\*\* Printer Test End \*\*\*\*\*

 #4714

Operator Signature  
 BRADLEY WINGERTER

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005948  
Location = GRAF      8164.13.00 06/09  
06/24/2016      14:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:38
02 Std. Gas	0.078	14:38
03 Room Air	0.000	14:39
04 Std. Gas	0.078	14:39
05 Room Air	0.000	14:40
06 Std. Gas	0.078	14:40
07 Room Air	0.000	14:41

Lot No = 01316080A2

Cyl No = 27

Exp Date = 02/05/2018

County = 50

Oper No. = 130890



Operator Signature  
BRADLEY WINGERTER

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

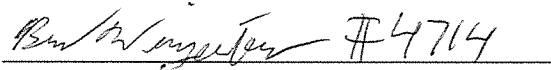
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005948  
Location = GRAF      8164.13.00 06/09  
06/24/2016      14:57

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	14:58
02 Room Air	0.000	14:59

\*Invalid Test  
Inhibited - RFI

Lot No = 01316080A2  
Cyl No = 27  
Exp Date = 02/05/2018  
County = 50      Oper No. = 130890

 #4714

Operator Signature  
BRADLEY WINGERTER

Remarks: