



# ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Rebecca Kopp</i>		
Location <i>Burleigh County Sheriff's Dept</i>	Intoxilyzer® Serial Number <i>80-005952</i>	
Gas Lot Number <i>09216080A3</i>	Gas Cylinder Number <i>31</i>	Gas Expiration Date <i>5-5-18</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. 0. 082 AC
  - B. 0. 081 AC
  - C. 0. 081 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>Rebecca Kopp</i>	Date <i>6-27-16</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>29 June 16</i>


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005952  
Location = BCSO                8164.13.00 06/09  
06/27/2016                        10:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:26
02 Std. Gas	0.082	10:26
03 Room Air	0.000	10:26
04 Std. Gas	0.081	10:27
05 Room Air	0.000	10:27
06 Std. Gas	0.081	10:28
07 Room Air	0.000	10:28

Lot No = 09216080A3  
Cyl No = 31  
Exp Date = 05/05/2018  
County = 08                      Oper No. = 041005

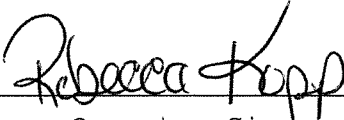

Operator Signature  
REBECCA KOPP

Remarks:

Form 106-I8000

New Cylinder Install