



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (03-2014)

Serial Number 80-006496	Instrument Location Bowman Sheriff's Office
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000. *N/A*
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *N/A*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Frank Thale</i>	Date <i>6-27-2016</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Shenaver</i>	Date <i>27 June 16</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Eden* Certified Date: *14 Nov. 2016*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006496
Location = BOWM 8164.13.00 06/09
06/27/2016 12:29

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 01316080A2
Standard Cyl #? 25
Standard Expiration? 02/05/2018
Oper No? 130015

Flow Cal. Date: 08/18/2015
Slope 673
Intercept -654167

IR Calibration Date: 08/18/2015
3um 9um

	3um	9um
0th Coef(*100):	-22081	-15963
1st Coef(*100):	278490	136818
2nd Coef(*100):	2086	1282
H2O adj(mg/l*10k):	598	386

***** Printer Test End *****



Operator Signature
FRANK EBERLE

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006496
Location = BOWM 8164.13.00 06/09
06/27/2016 12:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:31
02 Std. Gas	0.078	12:31
03 Room Air	0.000	12:32
04 Std. Gas	0.078	12:32
05 Room Air	0.000	12:33
06 Std. Gas	0.079	12:33
07 Room Air	0.000	12:34

Lot No = 01316080A2
Cyl No = 25
Exp Date = 02/05/2018
County = 06 Oper No. = 130015

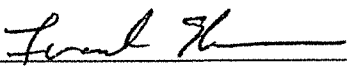

Operator Signature
FRANK EBERLE

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006496
Location = BOWM 8164.13.00 06/09
06/27/2016 12:38

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	12:39
02 Room Air	0.000	12:39

*Invalid Test
Inhibited - RFI

Lot No = 01316080A2
Cyl No = 25
Exp Date = 02/05/2018
County = 06 Oper No. = 130015

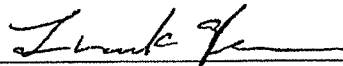

Operator Signature
FRANK EBERLE

Remarks:

Form 106-I8000