



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (1-2010)

Serial Number: 80-004206 Instrument Location: Richland County Jail

Reason for Install/Repair:

Install After Lab Repair/Inspection

Other (Specify) _____

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level 1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: _____

Sgt. Mark Sub
 Field Inspector Signature

6/30/10
 Date

Deb Kashner
 Reviewed By

06 July 2010
 Date

80-004206
 P. 100 of 103
 FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = WAHP 8164.13.00 06/09
06/30/2010 23:50

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.077
Standard Lot #? 659358
Standard Cyl #? 52
Standard Expiration? 09/03/2011
Oper No? 020401

Flow Cal. Date: 06/10/2009
Slope 690
Intercept -645883

IR Calibration Date: 06/07/2010

	3um	9um
0th Coef(*100):	-17579	-21810
1st Coef(*100):	254057	127149
2nd Coef(*100):	4591	1650
H2O adj(mg/l*10k):	756	795

***** Printer Test End *****



Operator Signature
MATTHEW W. ANDERSON

Remarks:

Good test


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = WAHP 8164.13.00 06/09
06/30/2010 23:50

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	23:51
02 Std. Gas	0.080	23:51
03 Room Air	0.000	23:52
04 Std. Gas	0.080	23:52
05 Room Air	0.000	23:53
06 Std. Gas	0.080	23:53
07 Room Air	0.000	23:54

Lot No = 659358
Cyl No = 52
Exp Date = 09/03/2011
County = 39 Oper No. = 020401

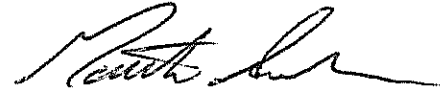

Operator Signature
MATTHEW W. ANDERSON

Remarks:

Good test 2nd ACA

Form 106-I8000

*80-004206
P. 102 of 103
JAN*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = WAHP 8164.13.00 06/09
06/30/2010 23:54

Test	AC	Time
01 Diagnostic	OK	23:56
02 Room Air	RFI*	23:56
03 Room Air	0.000	23:56

*Invalid Test
Inhibited - RFI

Sub Name = RFI, TEST NMN
Sub DOB = 03/20/1979
Sub Sex = Male Weight = 230
Test = DUI Cit = 22222222
Dr. Lic. = ND/QWE343456
Lot No = 659358
Cyl No = 52
Expiration Date = 09/03/2011
County = 39 Oper No. = 020401

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
MATTHEW W. ANDERSON

Remarks:

Form 106-I8000

80-004206
P. 103 of 103
jam