



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (1-2010)

Serial Number: 80-004939 Instrument Location: MAND.

Reason for Install/Repair:

Install After Lab Repair/Inspection

Other (Specify) NEW INSTALL

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level 1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: \_\_\_\_\_

*[Handwritten Signature]*

Field Inspector Signature

Date

07-01-10

*[Handwritten Signature: Deb Kashner]*

Reviewed By

Date

07 July 2010

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000       SN 80-004939  
Location = TOXL               8164.13.00 06/09  
07/01/2010                       10:06

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:           Enabled  
Start Test Sequence:       DABACABA  
Display Prelim Rslt?       Yes  
Display Third Digit?       Yes  
Inhib Printer(Y/N)?       No  
Display Volume?           No  
Disable On Memfull?       Yes  
# of Print Copies?         1  
Select Std (D/W/I)?       Dry  
Standard Value?           0.080  
Standard Lot #?           659358  
Standard Cyl #?           40  
Standard Expiration?       09/03/2011  
Oper No?                   068019

Flow Cal. Date:           01/18/2010  
Slope                      724  
Intercept                 -779308

IR Calibration Date:       03/10/2010

	3um	9um
0th Coef(*100):	-45908	-35439
1st Coef(*100):	263128	135479
2nd Coef(*100):	1658	1038
H2O adj(mg/l*10k):	832	527

\*\*\*\*\* Printer Test End \*\*\*\*\*

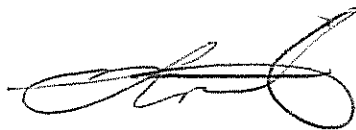

Operator Signature  
LORI E. FLATEN

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004939  
Location = TOXL      8164.13.00 06/09  
07/01/2010      10:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:08
02 Std. Gas	0.078	10:08
03 Room Air	0.000	10:08
04 Std. Gas	0.079	10:09
05 Room Air	0.000	10:09
06 Std. Gas	0.079	10:10
07 Room Air	0.000	10:10


Lot No = 659358

Cyl No = 40

Exp Date = 09/03/2011

County = 30

Oper No. = 068019

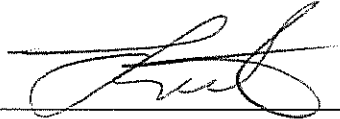

Operator Signature  
LORI E. FLATEN

Remarks:

Form 106-18000

80-004939  
P. 414/42 JEM


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004939  
Location = TOXL      8164.13.00 06/09  
07/01/2010      10:13

Test	AC	Time
01 Room Air	RFI*	10:15
02 Room Air	0.000	10:15

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, TEST NA  
Sub DOB = 01/01/1957  
Sub Sex = Female      Weight = 123  
Test = OTH      Cit = 23445  
Dr. Lic. = ND/34556  
Lot No = 659358  
Cyl No = 40  
Expiration Date = 09/03/2011  
County = 30      Oper No. = 068019

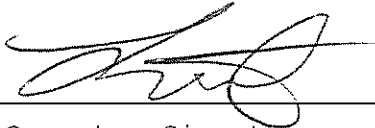

Operator Signature  
LORI E. FLATEN

Remarks:

Form 106-18000

80-004939  
P. 42 of 42  
JEM