



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (5-2011)

Serial Number 80-004938	Instrument Location SBCP
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P). ✓
 - B. ACA Test (Level 1, Function C). ✓
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air). ✓
- 7. Repair and/or Maintenance Performed (if Any): None
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date 07-04-2013
Reviewed By (Crime Laboratory Use Only) <i>Deb Kasher</i>	Date 16 July 13

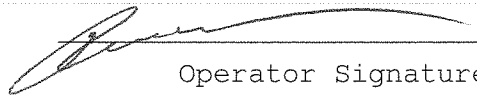
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = SBCP 8164.13.00 06/09
07/04/2013 15:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:03
02 Std. Gas	0.078	15:04
03 Room Air	0.000	15:04
04 Std. Gas	0.077	15:05
05 Room Air	0.000	15:05
06 Std. Gas	0.078	15:05
07 Room Air	0.000	15:06

Lot No = 16512080A2
Cyl No = 30
Exp Date = 08/01/2014
County = 46 Oper No. = 011017


Operator Signature
JUSTIN WALLERICH

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = SBCP 8164.13.00 06/09
07/04/2013 15:06

Test	AC	Time
01 Diagnostic	OK	15:08
02 Room Air	0.000	15:08
03 *Subject Test	RFI*	15:08
04 Room Air	0.000	15:09

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST NA
Sub DOB = 11/20/1995
Sub Sex = Male Weight = 222
Test = DUI Cit = 1111
Dr. Lic. = ND/NA
Lot No = 16512080A2
Cyl No = 30
Expiration Date = 08/01/2014
County = 46 Oper No. = 011017

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
JUSTIN WALLERICH

Remarks: