



ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Tracy L. Meidinger</i>		
Location <i>UND-PO</i>	Intoxilyzer® Serial Number <i>80-005357</i>	
Gas Lot Number <i>19615080A5</i>	Gas Cylinder Number <i>21</i>	Gas Expiration Date <i>09/05/2017</i>

Check When Done:

1. Scan/Enter Gas Cylinder Information (Level 1, Function S).

2. Perform an ACA Test (Level 1, Function C).

Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).

A. 0. 079 AC

B. 0. 078 AC

C. 0. 078 AC

3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.

4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.

5. Send the Following to the Crime Laboratory:

A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).

B. ACA Test Record.

Chemical Test Operator Signature <i>Tracy L. Meidinger</i>	Date <i>7-05-2016</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>08 July 16</i>

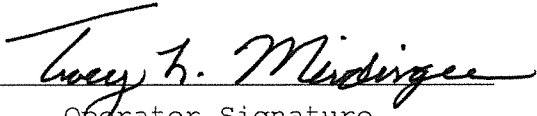
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005357
Location = UNDP 8164.13.00 06/09
07/05/2016 12:46

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:46
02 Std. Gas	0.079	12:47
03 Room Air	0.000	12:47
04 Std. Gas	0.078	12:48
05 Room Air	0.000	12:48
06 Std. Gas	0.078	12:49
07 Room Air	0.000	12:49

Lot No = 19615080A5
Cyl No = 21
Exp Date = 09/05/2017
County = 18 Oper No. = 119409


Operator Signature
TRACY MEIDINGER

Remarks: