



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (5-2011)

Serial Number <i>80-004938</i>	Instrument Location <i>Coop (Eggerstown)</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P). ✓
 - B. ACA Test (Level 1, Function C). ✓
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air). ✓
- 7. Repair and/or Maintenance Performed (if Any): *None*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>07-08-13</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kasher</i>	Date <i>16 July 13</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Eden* Certified Date: *23 Nov. 2016*

Intoxilyzer Test Record and Checklist
 NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
 North Dakota Model 8000 SN 80-004938
 Location = COOP 8164.13.00 06/09
 07/08/2013 11:00

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
 Start Test Sequence: DABACABA
 Display Prelim Rslt? Yes
 Display Third Digit? Yes
 Inhib Printer(Y/N)? No
 Display Volume? No
 Disable On Memfull? Yes
 # of Print Copies? 1
 Select Std (D/W/I)? Dry
 Standard Value? 0.080
 Standard Lot #? 16512080A2
 Standard Cyl #? 30
 Standard Expiration? 08/01/2014
 Oper No? 011017

Flow Cal. Date: 01/25/2010
 Slope 712
 Intercept -604165

IR Calibration Date: 05/02/2013

	3um	9um
0th Coef(*100):	-32801	-32122
1st Coef(*100):	279293	140886
2nd Coef(*100):	-301	554
H2O adj(mg/l*10k):	660	569

***** Printer Test End *****

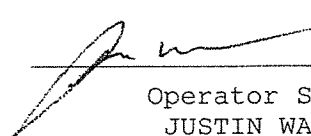

 Operator Signature
 JUSTIN WALLERICH

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = COOP 8164.13.00 06/09
07/08/2013 11:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:01
02 Std. Gas	0.079	11:01
03 Room Air	0.000	11:02
04 Std. Gas	0.078	11:02
05 Room Air	0.000	11:03
06 Std. Gas	0.078	11:03
07 Room Air	0.000	11:03

Lot No = 16512080A2
Cyl No = 30
Exp Date = 08/01/2014
County = 20 Oper No. = 011017

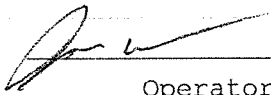

Operator Signature
JUSTIN WALLERICH

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = COOP 8164.13.00 06/09
07/08/2013 11:09

Test	AC	Time
01 Diagnostic	OK	11:10
02 Room Air	0.000	11:10
03 *Subject Test	RFI*	11:11
04 Room Air	0.000	11:11

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST TEST
Sub DOB = 11/22/1978
Sub Sex = Male Weight = 200
Test = DUI Cit = 1
Dr. Lic. = ND/NA
Lot No = 16512080A2
Cyl No = 30
Expiration Date = 08/01/2014
County = 20 Oper No. = 011017

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
JUSTIN WALLERICH

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = COOP 8164.13.00 06/09
07/08/2013 11:11

Test	AC	Time
01 Diagnostic	OK	11:12
02 Room Air	RFI*	11:13
03 Room Air	0.000	11:13

*Invalid Test
Inhibited - RFI

Sub Name = TEST , NA NA
Sub DOB = 11/21/1984
Sub Sex = Male Weight = 200
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 16512080A2
Cyl No = 30
Expiration Date = 08/01/2014
County = 20 Oper No. = 011017

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

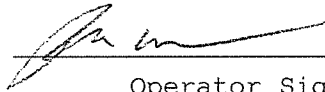

Operator Signature
JUSTIN WALLERICH

Remarks: