



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (5-2011)

Serial Number <i>80-004935</i>	Instrument Location <i>Fort Yates Police Dept.</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P). ✓
 - B. ACA Test (Level 1, Function C). ✓
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air). ✓
- NA* 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Steele Reed</i>	Date <i>7-11-11</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <i>18 July 2011</i>

*80-004935
P. 96 + 99
JFM*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = FYPD 8164.13.00 06/09
07/11/2011 15:54

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 15410080A1
Standard Cyl #? 34
Standard Expiration? 07/01/2012
Oper No? 020717

Flow Cal. Date: 01/20/2010
Slope 696
Intercept -664812

IR Calibration Date: 06/29/2011
 3um 9um

	3um	9um
0th Coef(*100):	1727	15
1st Coef(*100):	251148	124082
2nd Coef(*100):	6620	2476
H2O adj(mg/l*10k):	534	554

***** Printer Test End *****

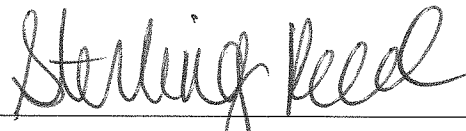

Operator Signature
STERLING R. REED

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = FYPD 8164.13.00 06/09
07/11/2011 15:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:55
02 Std. Gas	0.079	15:56
03 Room Air	0.000	15:56
04 Std. Gas	0.079	15:57
05 Room Air	0.000	15:57
06 Std. Gas	0.079	15:57
07 Room Air	0.000	15:58

Lot No = 15410080A1
Cyl No = 34
Exp Date = 07/01/2012
County = 61 Oper No. = 020717

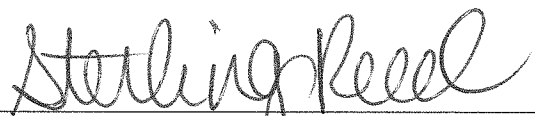

Operator Signature
STERLING R. REED

Remarks:

Form 106-I8000

80-004935
P. 98 of 99
VAM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = FYPD 8164.13.00 06/09
07/11/2011 15:59

Test	AC	Time
01 Diagnostic	OK	16:00
02 Room Air	RFI*	16:00
03 Room Air	0.000	16:01

*Invalid Test
Inhibited - RFI

Sub Name = JONES, JIM J
Sub DOB = 11/11/1975
Sub Sex = Male Weight = 200
Test = DUI Cit = 0001
Dr. Lic. = ND/JON759801
Lot No = 15410080A1
Cyl No = 34
Expiration Date = 07/01/2012
County = 61 Oper No. = 020717

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

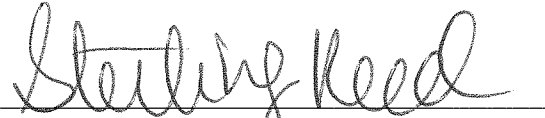

Operator Signature
STERLING R. REED

Remarks:

Form 106-I8000

80-004935
P. 99 of 99
RTR