



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (03-2014)

Serial Number <b>80-006501</b>	Instrument Location <b>GRAND FORKS COUNTY CORRECTIONAL CENTER</b>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u><b>GAS CYLINDER CHANGE</b></u>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): **GAS CYLINDER CHANGE**
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <b>7/12/2016</b> ✓
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <b>13 July 16</b>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *[Signature]*    Certified Date: **10 Nov. 16**




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006501  
Location = GFSO      8164.13.00 06/09  
07/12/2016      19:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	19:36
02 Std. Gas	0.079	19:36
03 Room Air	0.000	19:37
04 Std. Gas	0.079	19:37
05 Room Air	0.000	19:38
06 Std. Gas	0.080	19:38
07 Room Air	0.000	19:39

Lot No = 09216080A3  
Cyl No = 34  
Exp Date = 05/05/2018  
County = 18      Oper No. = 020836

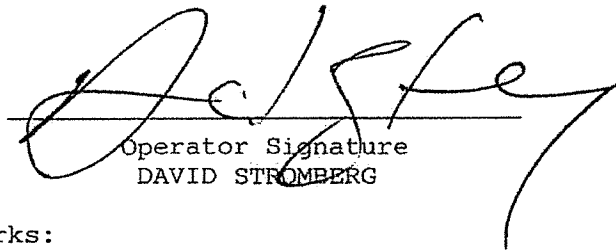  
Operator Signature  
DAVID STROMBERG

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006501  
Location = GFSO      8164.13.00 06/09  
07/12/2016      19:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	19:40
02 Room Air	0.000	19:40

\*Invalid Test  
Inhibited - RFI

Lot No = 09216080A3  
Cyl No = 34  
Exp Date = 05/05/2018  
County = 18      Oper No. = 020836

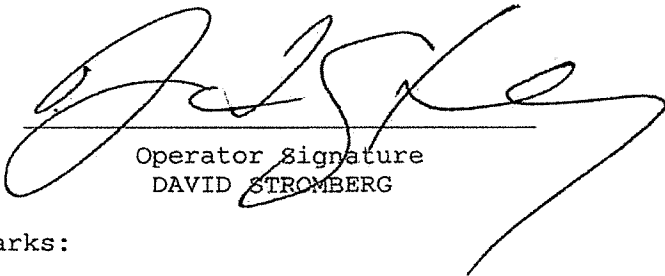

Operator Signature  
DAVID STROMBERG

Remarks: