



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (5-2011)

Serial Number <i>80-003057</i>	Instrument Location <i>Adams Cu So HETT</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>J. M. Walker</i>	Date <i>7-13-11</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kasher</i>	Date <i>18 July 2011</i>

80-003057
P. 144 of 147
ym

Intoxilyzer Test Record and Checklist
 NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
 North Dakota Model 8000 SN 80-003057
 Location = HETT 8164.13.00 06/09
 07/13/2011 09:43

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
 Start Test Sequence: DABACABA
 Display Prelim Rslt? Yes
 Display Third Digit? Yes
 Inhib Printer(Y/N)? No
 Display Volume? No
 Disable On Memfull? Yes
 # of Print Copies? 1
 Select Std (D/W/I)? Dry
 Standard Value? 0.080
 Standard Lot #? 11611080A2
 Standard Cyl #? 112
 Standard Expiration? 06/01/2013
 Oper No? 049130

Flow Cal. Date: 11/29/2010
 Slope 653
 Intercept -334316

IR Calibration Date: 06/14/2011
 3um 9um

	3um	9um
0th Coef(*100):	-6449	357
1st Coef(*100):	246728	129732
2nd Coef(*100):	5256	2318
H2O adj(mg/l*10k):	576	529

***** Printer Test End *****



 Operator Signature
 EUGENE M. MOLBERT

Remarks:

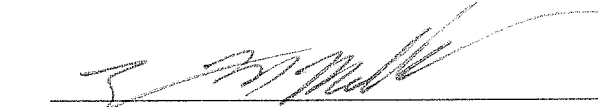
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = HETT 8164.13.00 06/09
07/13/2011 09:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:44
02 Std. Gas	0.080	09:45
03 Room Air	0.000	09:45
04 Std. Gas	0.080	09:46
05 Room Air	0.000	09:46
06 Std. Gas	0.081	09:46
07 Room Air	0.000	09:47

Lot No = 11611080A2
Cyl No = 112
Exp Date = 06/01/2013
County = 01 Oper No. = 049130


Operator Signature
EUGENE M. MOLBERT

Remarks:

Form 106-I8000

80-003057
P. 146 of 147
JTM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = HETT 8164.13.00 06/09
07/13/2011 09:48

Test	AC	Time
01 Diagnostic	OK	09:50
02 Room Air	RFI*	09:50
03 Room Air	RFI*	09:51

*Invalid Test
Inhibited - RFI

Sub Name = TEST, SUBJECT NA
Sub DOB = 12/12/1992
Sub Sex = Male Weight = 180
Test = DUI Cit = 12334
Dr. Lic. = ND/113414
Lot No = 11611080A2
Cyl No = 112
Expiration Date = 06/01/2013
County = 01 Oper No. = 049130

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.


Operator Signature
EUGENE M. MOLBERT

Remarks:

Form 106-I8000

80-003057
P.147 of 147
EM