



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59281 (03-2014)

Serial Number <i>80-005950</i>	Instrument Location <i>ETOT</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

### Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Jason DeMauro</i>	Date <i>7-25-14</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashur</i>	Date <i>15 Sept 14</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005950  
Location = FTOT                    8164.13.00 06/09  
07/25/2014                            20:47

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^\*()\_+?

Current Instrument Setup

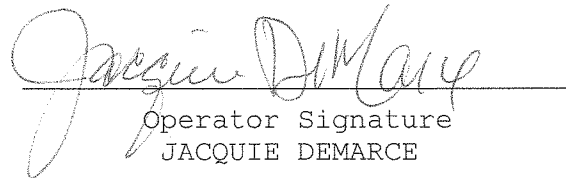
Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            33913080A4  
Standard Cyl #?            46  
Standard Expiration?        01/01/2016  
Oper No?                    100103

Flow Cal. Date:            06/20/2013  
Slope                        713  
Intercept                   -723515

IR Calibration Date:        06/20/2013  
                                    3um                    9um

-----  
0th Coef(\*100):            -37081                    -23816  
1st Coef(\*100):            290549                    133411  
2nd Coef(\*100):            552                        1090  
H2O adj(mg/l\*10k):        683                        502

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
Operator Signature  
JACQUIE DEMARCE

Remarks:

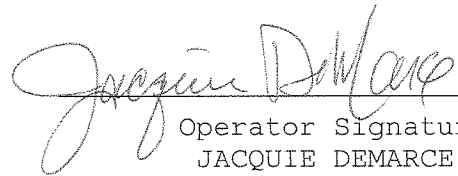
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005950  
Location = FTOT                    8164.13.00 06/09  
07/25/2014                            20:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	20:55
02 Std. Gas	0.081	20:55
03 Room Air	0.000	20:56
04 Std. Gas	0.081	20:56
05 Room Air	0.000	20:57
06 Std. Gas	0.081	20:57
07 Room Air	0.000	20:58

Lot No = 33913080A4  
Cyl No = 46  
Exp Date = 01/01/2016  
County = 03                            Oper No. = 100103

  
Operator Signature  
JACQUIE DEMARCE

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

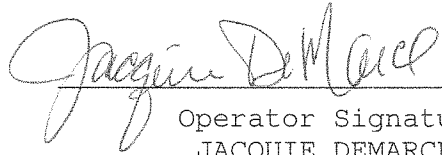
CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005950  
Location = FTOT                 8164.13.00 06/09  
07/25/2014                        20:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	20:53
02 Std. Gas	0.081	20:53
03 Room Air	RFI*	20:54
04 Room Air	0.000	20:54

\*Invalid Test  
Inhibited - RFI

Lot No = 33913080A4  
Cyl No = 46  
Exp Date = 01/01/2016  
County = 03                            Oper No. = 100103

  
\_\_\_\_\_  
Operator Signature  
JACQUIE DEMARCE

Remarks: