

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004948

Location: TOXL

- | | Check When Done |
|---|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5144</u> Lot No. <u> </u> Rep AC <u>0.021</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5114</u> Lot No. <u> </u> Rep AC <u>0.298</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR5186</u> Lot No. <u> </u> Rep AC <u>0.100 AC</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>709352</u>	Cyl No. <u>38</u>	Exp. Date <u>12 Dec 2011</u>
Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.081</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.079</u>
Test 3 <u>0.079</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

L. Tank pressure, Level 3 Func D Sub G P: 466 psi Reg 500 psi

M. Remarks/Maintenance record: Annual inspection

26 July 2010
Date

Deb Kashur
Field Inspector's Signature

29 July 2010
Date

Mary Bearn
Reviewed by

80-004948
P. 304/38
KML

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = TOXL 8164.13.00 06/09
07/26/2010 14:11

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 709352
Standard Cyl #? 38
Standard Expiration? 12/23/2011
Oper No? 777777

Flow Cal. Date: 07/14/2010
Slope 778
Intercept -795769

IR Calibration Date: 07/14/2010

	3um	9um
0th Coef(*100):	-31061	-21991
1st Coef(*100):	265097	133596
2nd Coef(*100):	802	1052
H2O adj(mg/l*10k):	675	417

***** Printer Test End *****

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Print Test

Form 106-I8000

80-004948
P. 31 of 38
JKH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = TOXL 8164.13.00 06/09
07/26/2010 14:13

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:14
02 Std. Sol.	0.020	14:14
03 Room Air	0.000	14:15
04 Std. Sol.	0.019	14:16
05 Room Air	0.000	14:16
06 Std. Sol.	0.020	14:17
07 Room Air	0.000	14:17

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Low AC 0.021AC

Form 106-I8000

80-004948
P. 32 of 38
RTR

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = TOXL 8164.13.00 06/09
07/26/2010 14:19

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:20
02 Std. Sol.	0.292	14:20
03 Room Air	0.000	14:21
04 Std. Sol.	0.293	14:22
05 Room Air	0.000	14:22
06 Std. Sol.	0.293	14:23
07 Room Air	0.000	14:23

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 77777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.298 AC

Form 106-I8000

80-004948
P. 33 of 3P
FDM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = TOXL 8164.13.00 06/09
07/26/2010 14:25

Test	AC	Time
01 Room Air	0.000	14:26
02 *Subject Test	INT*	14:26
03 Room Air	0.000	14:27

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = 00000
Std Sol No = 000
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.100AC + acetone

Form 106-I8000

80-004948
P. 3 of 3P
JK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = TOXL 8164.13.00 06/09
07/26/2010 14:28

Test	AC	Time
01 Diagnostic	OK	14:29
02 Room Air	RFI*	14:29
03 Room Air	0.000	14:29

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 709352
Cyl No = 38
Expiration Date = 12/23/2011
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

80-004948
P. 35-138
KEM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = TOXL 8164.13.00 06/09
07/26/2010 15:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:11
02 Std. Gas	0.080	15:12
03 Room Air	0.000	15:12
04 Std. Gas	0.079	15:13
05 Room Air	0.000	15:13
06 Std. Gas	0.079	15:13
07 Room Air	0.000	15:14

Lot No = 709352
Cyl No = 38
Exp Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

*80-004948
P. 36 of 38
KLM*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = TOXL 8164.13.00 06/09
07/26/2010 15:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:15
02 Std. Gas	0.080	15:15
03 Room Air	0.000	15:15
04 Std. Gas	0.079	15:16
05 Room Air	0.000	15:16
06 Std. Gas	0.080	15:17
07 Room Air	0.000	15:17

Lot No = 709352
Cyl No = 38
Exp Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

*80-004948
P. 37.438
FM*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = TOXL 8164.13.00 06/09
07/26/2010 15:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:26
02 Std. Gas	0.081	15:26
03 Room Air	0.000	15:27
04 Std. Gas	0.079	15:27
05 Room Air	0.000	15:28
06 Std. Gas	0.080	15:28
07 Room Air	0.000	15:28

Lot No = 709352
Cyl No = 38
Exp Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

*80-004948
P. 38 of 38
KAK*