



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (03-2014)

Serial Number <u>80-004191</u>	Instrument Location <u>Valley City</u>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): NA
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date <u>7-25-14</u>
Reviewed By (Crime Laboratory Use Only) <u>Deb Kashner</u>	Date <u>28 July 14</u>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: Charles E. Eds Certified Date: 07 Nov. 16




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004191  
Location = VALL      8164.13.00 06/09  
07/25/2014      16:50

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:50
02 Std. Gas	0.078	16:51
03 Room Air	0.000	16:51
04 Std. Gas	0.078	16:51
05 Room Air	0.000	16:52
06 Std. Gas	0.077	16:52
07 Room Air	0.000	16:53

Lot No = 16512080A2  
Cyl No = 47  
Exp. Date = 08/01/2014  
County = 02      Oper No. = 130409

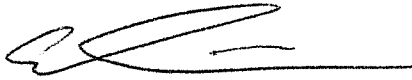

Operator Signature  
ANDREW EISENSCHENK

Remarks: *INSTALL*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004191  
Location = VALL      8164.13.00 06/09  
07/25/2014      16:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	16:49
02 Room Air	0.000	16:49

\*Invalid Test  
Inhibited - RFI

Lot No = 16512080A2  
Cyl No = 47  
Exp Date = 08/01/2014  
County = 02      Oper No. = 130409

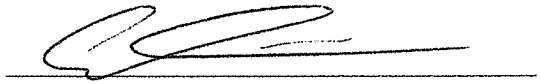

Operator Signature  
ANDREW EISENSCHENK

Remarks: RADIO IN ROOM  
RE-TAKE ACA

Form 106-I8000

INSTALL