



**ETHANOL BREATH STANDARD CYLINDER REPORT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59282 (5-2011)

Chemical Test Operator Name (Print) <i>TOM HERZIG</i>		
Location <i>JAME STEUBEN COUNTY CORRECTIONS</i>	Intoxilyzer® Serial Number <i>90-005361</i>	
Gas Lot Number <i>339130 80A4</i>	Gas Cylinder Number <i>17</i>	Gas Expiration Date <i>01-01-2016</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. 0. 079 AC  
 B. 0. 079 AC  
 C. 0. 079 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Breath Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>08-04-14</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <i>12 Aug 2014</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005361  
Location = JAME      8164.13.00 06/09  
08/04/2014      10:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:34
02 Std. Gas	0.079	10:34
03 Room Air	0.000	10:35
04 Std. Gas	0.079	10:35
05 Room Air	0.000	10:36
06 Std. Gas	0.079	10:36
07 Room Air	RFI*	10:36
08 Room Air	0.000	10:37

\*Invalid Test  
Inhibited - RFI

Lot No = 33913080A4  
Cyl No = 17  
Exp Date = 01/01/2016  
County = 47      Oper No. = 039706

  
Operator Signature  
THOMAS HERZIG

Remarks:

Form 106-I8000

*INSTALL OF NEW CYLINDER*

*This ACA will need to be rerun because of the RFI during the test. Deb Kashner*