

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004184

Location: TOXL

- | | Check When Done |
|--|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5144</u> Lot No. <u> </u> Rep AC <u>0.021</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5114</u> Lot No. <u> </u> Rep AC <u>0.298</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR5186</u> Lot No. <u> </u> Rep AC <u>0.106</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>709352</u>	Cyl No. <u>38</u>	Exp. Date <u>12/23/2011</u>
Test 1 <u>0.079</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

L. Tank pressure, Level 3 Func D Sub G P: 462 psi Reg 480 psi

M. Remarks/Maintenance record: annual inspection

10 Aug 2010
Date

Deb Kashner
Field Inspector's Signature

10 Aug 2010
Date

Mary Plarion
Reviewed by

80-004184
p. 86 of 94
JAN

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.13.00 06/09
08/10/2010 09:34

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 709352
Standard Cyl #? 38
Standard Expiration? 12/23/2011
Oper No? 777777

Flow Cal. Date: 07/30/2010
Slope 804
Intercept -925008

IR Calibration Date: 07/30/2010

	3um	9um
0th Coef(*100):	-19612	-18975
1st Coef(*100):	270159	137378
2nd Coef(*100):	1174	963
H2O adj(mg/l*10k):	427	372

***** Printer Test End *****

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Print test

Form 106-I8000

80-009184
P. 87 of 94
vth

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.13.00 06/09
08/10/2010 08:42

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:42
02 Std. Sol.	0.018	08:43
03 Room Air	0.000	08:44
04 Std. Sol.	0.019	08:44
05 Room Air	0.000	08:45
06 Std. Sol.	0.020	08:46
07 Room Air	0.000	08:46

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Low Ac 0.021AC

Form 106-I8000

*80-004184
P. 88 of 94
KA*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.13.00 06/09
08/10/2010 08:47

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:48
02 Std. Sol.	0.294	08:49
03 Room Air	0.000	08:49
04 Std. Sol.	0.294	08:50
05 Room Air	0.000	08:51
06 Std. Sol.	0.294	08:51
07 Room Air	0.000	08:52

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.298 AC

Form 106-18000

*80-004184
P. 89 of 94
AM*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.13.00 06/09
08/10/2010 09:35

Test	AC	Time
01 Room Air	0.000	09:36
02 *Subject Test	INT*	09:36
03 Room Air	0.000	09:37

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 709352
Cyl No = 38
Expiration Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.100 AC + acetone

Form 106-18000

80-004184
P. 90/94
VCR

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.13.00 06/09
08/10/2010 09:38

Test	AC	Time
01 Diagnostic	OK	09:39
02 Room Air	RFI*	09:39
03 Room Air	0.000	09:40

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 709352
Cyl No = 38
Expiration Date = 12/23/2011
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

*80-004184
p. 91 of 94
JKM*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.13.00 06/09
08/10/2010 09:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:41
02 Std. Gas	0.079	09:41
03 Room Air	0.000	09:42
04 Std. Gas	0.080	09:42
05 Room Air	0.000	09:43
06 Std. Gas	0.080	09:43
07 Room Air	0.000	09:44

Lot No = 709352
Cyl No = 38
Exp Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

calibration check 0.080 AC

Form 106-I8000

*80-004184
p. 92 of 94
PK*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.13.00 06/09
08/10/2010 09:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:52
02 Std. Gas	0.080	09:52
03 Room Air	0.000	09:52
04 Std. Gas	0.080	09:53
05 Room Air	0.000	09:53
06 Std. Gas	0.080	09:54
07 Room Air	0.000	09:54

Lot No = 709352
Cyl No = 38
Exp Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

*80-004184
p. 93 of 94
YK*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.13.00 06/09
08/10/2010 09:57

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:58
02 Std. Gas	0.080	09:58
03 Room Air	0.000	09:59
04 Std. Gas	0.080	09:59
05 Room Air	0.000	10:00
06 Std. Gas	0.080	10:00
07 Room Air	0.000	10:01

Lot No = 709352
Cyl No = 38
Exp Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-004184
P. 94 of 94
FM