



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59281 (03-2014)

Serial Number	Instrument Location
80-005957	COOP
Reason for Install/Repair	
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

### Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date 08-12-14 ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashur</i>	Date 05 Sept 14

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005957  
Location = COOP                    8164.13.00 06/09  
08/12/2014                            11:26

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

Current Instrument Setup

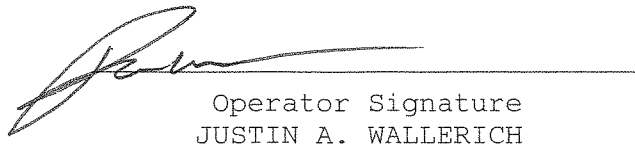
Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            13314080A2  
Standard Cyl #?            60  
Standard Expiration?       06/05/2016  
Oper No?                    011017

Flow Cal. Date:            06/21/2013  
Slope                       647  
Intercept                  -613317

IR Calibration Date:        04/09/2014

	3um	9um
0th Coef(*100):	-14149	-20695
1st Coef(*100):	278465	133145
2nd Coef(*100):	3010	1370
H2O adj(mg/l*10k):	673	615

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
Operator Signature  
JUSTIN A. WALLERICH

Remarks:


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005957  
Location = COOP      8164.13.00 06/09  
08/12/2014      11:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:28
02 Std. Gas	0.078	11:28
03 Room Air	0.000	11:29
04 Std. Gas	0.077	11:29
05 Room Air	0.000	11:30
06 Std. Gas	0.078	11:30
07 Room Air	0.000	11:30

Lot No = 13314080A2  
Cyl No = 60  
Exp Date = 06/05/2016  
County = 20      Oper No. = 011017

  
\_\_\_\_\_  
Operator Signature  
JUSTIN A. WALLERICH

Remarks:


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005957  
Location = COOP                8164.13.00 06/09  
08/12/2014                        19:13

Test	AC	Time
01 Room Air	0.000	19:14
02 *Subject Test	RFI*	19:15
03 Room Air	0.000	19:16

\*Invalid Test  
  Inhibited - RFI

Sub Name = TEST, TEST NA  
Sub DOB = 12/23/1987  
Sub Sex = Male                    Weight = 120  
Test = DUI                        Cit = 00  
Dr. Lic. = ND/NA  
Lot No = 13314080A2  
Cyl No = 60  
Expiration Date = 06/05/2016  
County = 20                        Oper No. = 011017

  
Operator Signature  
JUSTIN A. WALLERICH

Remarks: