



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (1-2010)

Serial Number: <u>80-004937</u>	Instrument Location: <u>Langdon, Cavalier County.</u>
Reason for Install/Repair: <input type="checkbox"/> Install After Lab Repair/Inspection <input checked="" type="checkbox"/> Other (Specify) <u>Replace unit</u>	

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: _____

Joe Knowler
Field Inspector Signature

8-17-2010
Date

Deb Kashner
Reviewed By

23 Aug 2010
Date

80-004937
P. 46 of 49
KM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004937
Location = LANG 8164.13.00 06/09
08/17/2010 13:18

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&*()_+?

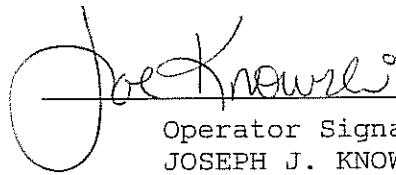
Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.075
Standard Lot #? 659358
Standard Cyl #? 22
Standard Expiration? 09/03/2011
Oper No? 109208

Flow Cal. Date: 01/18/2010
Slope 730
Intercept -779991

IR Calibration Date:	02/10/2010
	3um 9um
0th Coef(*100):	-23590 -32037
1st Coef(*100):	262965 133872
2nd Coef(*100):	-566 311
H2O adj(mg/l*10k):	569 537

***** Printer Test End *****


Operator Signature
JOSEPH J. KNOWSKI

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004937
Location = LANG 8164.13.00 06/09
08/17/2010 13:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:20
02 Std. Gas	0.077	13:20
03 Room Air	0.000	13:20
04 Std. Gas	0.077	13:21
05 Room Air	0.000	13:21
06 Std. Gas	0.076	13:22
07 Room Air	0.000	13:22

Lot No = 659358
Cyl No = 22
Exp Date = 09/03/2011
County = 10 Oper No. = 109208


Operator Signature
JOSEPH J. KNOWSKI

Remarks:

Form 106-I8000

80-004937
p. 48 of 49
Jm

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

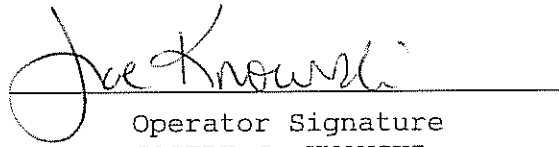
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004937
Location = LANG 8164.13.00 06/09
08/17/2010 13:23

Test	AC	Time
01 Diagnostic	OK	13:25
02 Room Air	RFI*	13:25
03 Room Air	0.000	13:25

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST NA
Sub DOB = 09/09/1990
Sub Sex = Male Weight = 220
Test = DUI Cit = 12345
Dr. Lic. = ND/TES907878
Lot No = 659358
Cyl No = 22
Expiration Date = 09/03/2011
County = 10 Oper No. = 109208

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.


Operator Signature
JOSEPH J. KNOWSKI

Remarks:

Form 106-I8000

80-00 4937
P. 49 of 49
JAN