



ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>MATTHEW ANDERSON</i>		
Location <i>RICHLAND COUNTY JAIL</i>	Intoxilyzer® Serial Number <i>80-005944</i>	
Gas Lot Number <i>33 913080 A4</i>	Gas Cylinder Number <i>20</i>	Gas Expiration Date <i>01/01/2016</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. *0.080* AC
 - B. *0.081* AC
 - C. *0.081* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Matthew Anderson</i>	Date <i>8/21/14</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashur</i>	Date <i>05 Sept 2014</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = WAHP 8164.13.00 06/09
08/21/2014 22:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	22:35
02 Std. Gas	0.080	22:35
03 Room Air	0.000	22:36
04 Std. Gas	0.081	22:36
05 Room Air	0.000	22:37
06 Std. Gas	0.081	22:37
07 Room Air	0.000	22:38

Lot No = 33913080A4
Cyl No = 20
Exp Date = 01/01/2016
County = 39 Oper No. = 020401



Operator Signature
MATTHEW ANDERSON

Remarks:

Cylinder report ACA Good test

Form 106-I8000