



ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (10-2013)

Chemical Test Operator Name (Print) Christopher S. Jenkins		
Location Stanley	Intoxilyzer® Serial Number 80-004948	
Gas Lot Number 13314080A2	Gas Cylinder Number 54	Gas Expiration Date 06/05/2016

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0. 080 AC
 - B. 0. 080 AC
 - C. 0. 079 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Christopher S. Jenkins</i>	Date 08/23/2014 ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date 05 Sept 14

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = STAN 8164.13.00 06/09
08/23/2014 22:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	22:33
02 Std. Gas	0.080	22:34
03 Room Air	0.000	22:34
04 Std. Gas	0.080	22:34
05 Room Air	0.000	22:35
06 Std. Gas	0.079	22:35
07 Room Air	0.000	22:36

Lot No = 13314080A2
Cyl No = 54
Exp Date = 06/05/2016
County = 31 Oper No. = 130248



Operator Signature
CHRISTOPHER JENKINS

Remarks: