



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (03-2014)

Serial Number <b>80-005947</b>	Instrument Location <b>Dickinson LEC</b>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): None
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date <b>8-23-16 ✓</b>
Reviewed By (Crime Laboratory Use Only) <b>Deb Shanaver</b>	Date <b>06 Sept 2016</b>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By:  Certified Date: **18 Nov. 2016**



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005947  
Location = DKPD      8164.13.00 06/09  
08/23/2016      17:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:20
02 Std. Gas	0.079	17:21
03 Room Air	0.000	17:21
04 Std. Gas	0.079	17:22
05 Room Air	0.000	17:22
06 Std. Gas	0.078	17:22
07 Room Air	0.000	17:23

Lot No = 29614080A2  
Cyl No = 10  
Exp Date = 12/05/2016  
County = 45      Oper No. = 010107



Operator Signature  
CHRISTOPHER MESSER

Remarks:

ACA Test

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005947  
Location = DKPD      8164.13.00 06/09  
08/23/2016      17:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:25
02 Std. Gas	0.078	17:25
03 Room Air	RFI*	17:25
04 Room Air	0.000	17:26

\*Invalid Test  
Inhibited - RFI

Lot No = 29614080A2  
Cyl No = 10  
Exp Date = 12/05/2016  
County = 45      Oper No. = 010107

  
Operator Signature  
CHRISTOPHER MESSER

Remarks:

*RFI Test*

Form 106-I8000