

*✓cm*

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-003058

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓  
Sim SN DR5190 Lot No. — Rep AC 0.021
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓  
Sim SN DR5113 Lot No. — Rep AC 0.297
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓  
Sim SN DR5135 Lot No. — Rep AC 0.10 AC + Acetone
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. 617774 Cyl No. 46 Exp. Date 5/28/2011  
Test 1 0.080 Test 4 0.080 Test 7 0.081  
Test 2 0.080 Test 5 0.079 Test 8 0.080  
Test 3 0.080 Test 6 0.080 Test 9 0.080  
Average 0.080

L. Remarks/Maintenance record: Annual Inspection  
25 August 2009 Mary Plou  
Date Field Inspector's Signature  
04 Sept 2009 Deb Keshur  
Date Reviewed by

08/09

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.12.00 06/09  
08/25/2009      16:50

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:      Enabled  
Start Test Sequence:      DABACABA  
Display Prelim Rslt?      Yes  
Display Third Digit?      Yes  
Inhib Printer(Y/N)?      No  
Display Volume?      No  
Disable On Memfull?      Yes  
# of Print Copies?      1  
Select Std (D/W/I)?      Dry  
Standard Value?      0.800  
Standard Lot #?      617774  
Standard Cyl #?      46  
Standard Expiration?      05/28/2011  
Oper No?      107501

Flow Cal. Date:      04/07/2008  
Slope      698  
Intercept      -678970

IR Calibration Date:      06/05/2008

	3um	9um
0th Coef(*100):	-18867	-13178
1st Coef(*100):	274682	135660
2nd Coef(*100):	1139	678
H2O adj(mg/l*10k):	434	216

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
MARGY PEARSON

Remarks:



Form 106-I8000

80-003058  
P. 2 of 9 File

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.12.00 06/09  
08/25/2009      11:33

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:33
02 Std. Sol.	0.018	11:34
03 Room Air	0.000	11:35
04 Std. Sol.	0.019	11:35
05 Room Air	0.000	11:36
06 Std. Sol.	0.019	11:37
07 Room Air	0.000	11:37


08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 107501

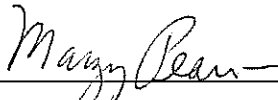

Operator Signature  
MARGY PEARSON

Remarks:

Form 106-I8000

Low AC 0.021

80-003058  
8-30-97

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.12.00 06/09  
08/25/2009      17:52

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:52
02 Std. Sol.	0.293	17:57
03 Room Air	0.000	17:58
04 Std. Sol.	0.308	17:58
05 Room Air	0.000	17:59
06 Std. Sol.	0.310	18:00
07 Room Air	0.000	18:00
08 Sim Temp = 34.0°C		

Simul Ser No = DR00000  
Std Sol No = 000  
County = 00      Oper No. = 107501

  
Operator Signature  
MARGY PEARSON

Remarks:

*Linearity*  
Form 106-I8000

80-003058  
P. 4 of 9 pm


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003058  
Location = TOXL                8164.12.00 06/09  
08/25/2009                        18:12

Test	AC	Time
01 Room Air	0.000	18:13
02 *Subject Test	INT*	18:13
03 Room Air	0.000	18:14

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 11/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Simul Ser No = DR00000  
Std Sol No = 000  
County = 08                            Oper No. = 107501

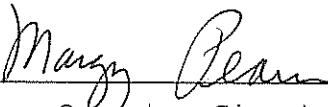  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

*Interferent*  
*Alc + Acetone*  
Form 106-I8000

80-003058  
P. 5 of 9

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.12.00 06/09  
08/25/2009      17:23

Test	AC	Time
01 Diagnostic	OK	17:24
02 Room Air	RFI*	17:24
03 Room Air	0.000	17:25

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 11/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 617774  
Cyl No = 46  
Expiration Date = 05/28/2011  
County = 08      Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-003058  
P. Coof 9/11/11

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.12.00 06/09  
08/25/2009      17:13

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:14
02 Std. Gas	0.081	17:14
03 Room Air	0.000	17:15
04 Std. Gas	0.080	17:15
05 Room Air	0.000	17:16
06 Std. Gas	0.080	17:16
07 Room Air	0.000	17:16


Lot No = 617774

Cyl No = 46

Exp Date = 05/28/2011

County = 08

Oper No. = 107501

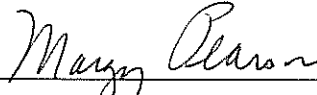

Operator Signature  
MARGY PEARSON

Remarks:

ACA

Form 106-18000

80-003058  
P. 2 of 9  
JFH

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.12.00 06/09  
08/25/2009      17:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:10
02 Std. Gas	0.080	17:10
03 Room Air	0.000	17:11
04 Std. Gas	0.079	17:11
05 Room Air	0.000	17:12
06 Std. Gas	0.080	17:12
07 Room Air	0.000	17:12


Lot No = 617774

Cyl No = 46

Exp Date = 05/28/2011

County = 08

Oper No. = 107501

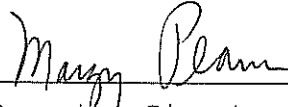

Operator Signature  
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-003058  
p. 8 of 9 JH



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.12.00 06/09  
08/25/2009      17:06

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:06
02 Std. Gas	0.080	17:07
03 Room Air	0.000	17:07
04 Std. Gas	0.080	17:08
05 Room Air	0.000	17:08
06 Std. Gas	0.080	17:08
07 Room Air	0.000	17:09


Lot No = 617774

Cyl No = 46

Exp Date = 05/28/2011

County = 08

Oper No. = 107501

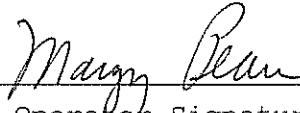

Operator Signature  
MARGY PEARSON

Remarks:

Form 106-I8000

ACA

80-003058  
p. 9 of 9 PM