

VCM

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-003067

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓
Sim SN DR 5190 Lot No. — Rep AC 0.021
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓
Sim SN DR 5113 Lot No. — Rep AC 0.297
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓
Sim SN DR 5135 Lot No. — Rep AC 0.10 AC + ACETONE
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. <u>617774</u>	Cyl No. <u>8</u>	Exp. Date <u>5/28/2011</u>
Test 1 <u>0.079</u>	Test 4 <u>0.080</u>	Test 7 <u>0.081</u>
Test 2 <u>0.080</u>	Test 5 <u>0.081</u>	Test 8 <u>0.080</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

L. Remarks/Maintenance record: Annual Inspection

<u>25 August 2009</u> Date	<u>Mary O'Leary</u> Field Inspector's Signature
<u>04 Sept 2009</u> Date	<u>Deb Kashur</u> Reviewed by

08/09

*80-003067
P. 1 of 9 FM*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.12.00 06/09
08/25/2009 17:56

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:56
02 Std. Sol.	0.019	17:58
03 Room Air	0.000	17:58
04 Std. Sol.	0.020	17:59
05 Room Air	0.000	17:59
06 Std. Sol.	0.019	18:00
07 Room Air	0.000	18:01


08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 0000

County = 08

Oper No. = 107501

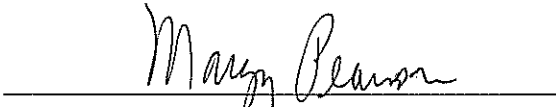

Operator Signature
MARGY PEARSON

Remarks:

LOW AC 0.021 AC

Form 106-I8000

80-003067
P. 3 of 9 PM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.12.00 06/09
08/25/2009 13:59


WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:00
02 Std. Sol.	0.292	14:01
03 Room Air	0.000	14:01
04 Std. Sol.	0.299	14:02
05 Room Air	0.000	14:02
06 Std. Sol.	0.298	14:03
07 Room Air	0.000	14:04

08 Sim Temp = 34.0°C

Simul Ser No = 000
Std Sol No = DR0000
County = 08

Oper No. = 107501

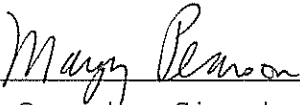

Operator Signature
MARGY PEARSON

Remarks:

Form 106-18000

Linearity

80-003067
P. 4 of 9
FM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.12.00 06/09
08/25/2009 18:14

Test	AC	Time
01 Room Air	0.000	18:16
02 *Subject Test	INT*	18:17
03 Room Air	0.000	18:17

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR0000
Std Sol No = 0000
County = 08 Oper No. = 107501

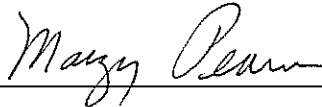

Operator Signature
MARGY PEARSON

Remarks: *Alcohol & Acetone*

Form 106-I8000

80-003067
P. Sol 9/11/11

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.12.00 06/09
08/25/2009 17:16

Test	AC	Time
01 Diagnostic	OK	17:17
02 Room Air	RFI*	17:17
03 Room Air	0.000	17:18

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 617774
Cyl No = 8
Expiration Date = 05/28/2011
County = 08 Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

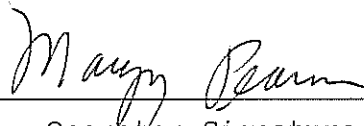

Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-003067
P. 6 of 9 file


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.12.00 06/09
08/25/2009 17:02

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:02
02 Std. Gas	0.079	17:03
03 Room Air	0.000	17:03
04 Std. Gas	0.080	17:04
05 Room Air	0.000	17:04
06 Std. Gas	0.081	17:04
07 Room Air	0.000	17:05

Lot No = 617774
Cyl No = 8
Exp Date = 05/28/2011
County = 08 Oper No. = 107501

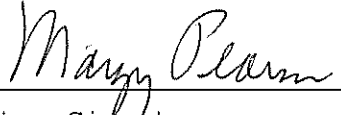

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-003067
P. 7 of 9 JRM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.12.00 06/09
08/25/2009 17:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:08
02 Std. Gas	0.080	17:08
03 Room Air	0.000	17:08
04 Std. Gas	0.081	17:09
05 Room Air	0.000	17:09
06 Std. Gas	0.081	17:10
07 Room Air	0.000	17:10

Lot No = 617774
Cyl No = 8
Exp Date = 05/28/2011
County = 08 Oper No. = 107501

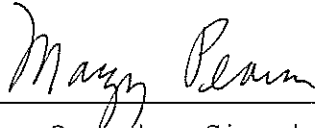

Operator Signature
MARGY PEARSON

Remarks:

Form 106-I8000

ACA

80-003067
P. 8 of 9


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.12.00 06/09
08/25/2009 17:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:11
02 Std. Gas	0.081	17:12
03 Room Air	0.000	17:12
04 Std. Gas	0.080	17:13
05 Room Air	0.000	17:13
06 Std. Gas	0.080	17:13
07 Room Air	0.000	17:14

Lot No = 617774
Cyl No = 8
Exp Date = 05/28/2011
County = 08 Oper No. = 107501

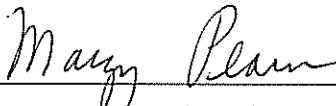

Operator Signature
MARGY PEARSON

Remarks:

Form 106-I8000

ACA

80-003067
P. 9 of 9 AL