

✓ cml

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-002669

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5190 Lot No. — Rep AC 0.021 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5113 Lot No. — Rep AC 0.297 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.) ✓
Sim SN DR5135 Lot No. — Rep AC 0.10% + Acetone
- J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. <u>617774</u>	Cyl No. <u>32</u>	Exp. Date <u>5/28/2011</u>
Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.082</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.080</u>
Test 3 <u>0.082</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>
Average <u>0.081</u>		

L. Remarks/Maintenance record: Annual Inspection

26 August 2009
Date

04 Sept 2009
Date

Margy Posner
Field Inspector's Signature

Deb Kashur
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.12.00 06/09
08/26/2009 15:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:15
02 Std. Sol.	0.019	15:16
03 Room Air	0.000	15:17
04 Std. Sol.	0.019	15:17
05 Room Air	0.000	15:18
06 Std. Sol.	0.020	15:19
07 Room Air	0.000	15:19


08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 000

County = 08

Oper No. = 107501

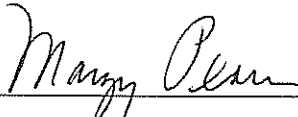

Operator Signature
MARGY PEARSON

Remarks:

LOW AC

Form 106-I8000

P. 3 of 9
SEM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.12.00 06/09
08/26/2009 16:45

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:46
02 Std. Sol.	0.287	16:47
03 Room Air	0.000	16:48
04 Std. Sol.	0.292	16:48
05 Room Air	0.000	16:49
06 Std. Sol.	0.291	16:50
07 Room Air	0.000	16:50

08 Sim Temp = 34.0°C

Simul Ser No = 000000
Std Sol No = 000
County = 08 Oper No. = 107501

Margy Pearson

Operator Signature
MARGY PEARSON

Remarks:

Linearity

Form 106-I8000

*P. 4 of 9
pm*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.12.00 06/09
08/26/2009 16:55

Test	AC	Time
01 Room Air	0.000	16:56
02 *Subject Test	INT*	16:56
03 Room Air	0.000	16:57

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = 000000
Std Sol No = 000
County = 08 Oper No. = 107501

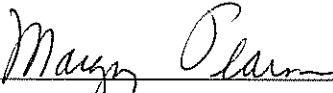

Operator Signature
MARGY PEARSON

Remarks:

Interfer

Alc + Acetone

Form 106-18000

*P. 5 of 9
JAN*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.12.00 06/09
08/26/2009 17:00

Test	AC	Time
01 Diagnostic	OK	17:01
02 Room Air	RFI*	17:01
03 Room Air	PUR**	17:02

*Invalid Test
 Inhibited - RFI
**Purge Fail

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = 000000
Std Sol No = 000
County = 08 Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

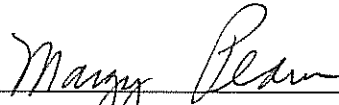

Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

P. Leaf 9
1/20


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.12.00 06/09
08/26/2009 15:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:05
02 Std. Gas	0.082	15:06
03 Room Air	0.000	15:06
04 Std. Gas	0.080	15:06
05 Room Air	0.000	15:07
06 Std. Gas	0.081	15:07
07 Room Air	0.000	15:08

Lot No = 617774
Cyl No = 32
Exp Date = 05/28/2011
County = 08 Oper No. = 107501

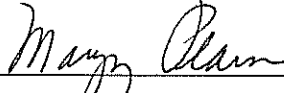

Operator Signature
MARGY PEARSON

Remarks:

AAA

Form 106-I8000

P. 7 of 9
12/26

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.12.00 06/09
08/26/2009 14:58

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:58
02 Std. Gas	0.081	14:59
03 Room Air	0.000	14:59
04 Std. Gas	0.081	15:00
05 Room Air	0.000	15:00
06 Std. Gas	0.082	15:00
07 Room Air	0.000	15:01

Lot No = 617774
Cyl No = 32
Exp Date = 05/28/2011
County = 08 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

P. 8 of 9
JAL


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.12.00 06/09
08/26/2009 15:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:02
02 Std. Gas	0.081	15:02
03 Room Air	0.000	15:03
04 Std. Gas	0.081	15:03
05 Room Air	0.000	15:04
06 Std. Gas	0.081	15:04
07 Room Air	0.000	15:04

Lot No = 617774
Cyl No = 32
Exp Date = 05/28/2011
County = 08 Oper No. = 107501

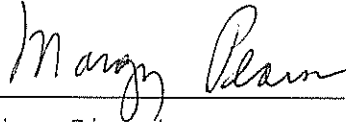

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

P. 9 of 9
pm