

*✓ CML*

**INTOXILYZER 5000 G and 8000 INSPECTION**

Serial No.: 80-003072

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓  
Sim SN DR5190 Lot No. — Rep AC 0.021
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓  
Sim SN DR5113 Lot No. — Rep AC 0.297
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. ✓  
(Attach test record.)  
Sim SN DR5135 Lot No. — Rep AC 0.102 ± Acetone
- J. RFI check. Run CMS mode. Key radio on first room air. ✓  
(Attach test record.)

K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. 617774 Cyl No. 42 Exp. Date 5/28/2011 <sup>in ATP</sup> 2011

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.079</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>

Average 0.081

L. Remarks/Maintenance record: Annual Inspection

26 August 2009  
Date

Mary Plummer  
Field Inspector's Signature

04 Sept 2009  
Date

Deb Kashner  
Reviewed by



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003072  
Location = TOXL      8164.12.00 06/09  
08/26/2009      16:47

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:48
02 Std. Sol.	0.017	16:49
03 Room Air	0.000	16:49
04 Std. Sol.	0.018	16:50
05 Room Air	0.000	16:51
06 Std. Sol.	0.018	16:51
07 Room Air	0.000	16:52

08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 00000

County = 08

Oper No. = 107501

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

*LOW AC*

Form 106-I8000

80-003072  
P. 3 of 9 FILE

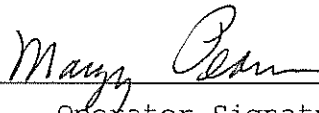
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003072  
Location = TOXL      8164.12.00 06/09  
08/26/2009      16:53

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:54
02 Std. Sol.	0.282	16:55
03 Room Air	0.000	16:55
04 Std. Sol.	0.286	16:56
05 Room Air	0.000	16:56
06 Std. Sol.	0.286	16:57
07 Room Air	0.000	16:58
08 Sim Temp = 34.0°C		

Simul Ser No = 000000  
Std Sol No = 00000  
County = 08      Oper No. = 107501

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:      *Linearity*

Form 106-I8000

80-003072  
P. 4 of 9 PK

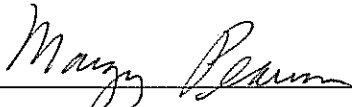
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003072  
Location = TOXL      8164.12.00 06/09  
08/26/2009      16:59

Test	AC	Time
01 Room Air	0.000	17:00
02 *Subject Test	INT*	17:00
03 Room Air	0.000	17:01

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 11/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS821456  
Simul Ser No = 000000  
Std Sol No = 00000  
County = 08      Oper No. = 107501

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

*Interferent*

Form 106-I8000

80-003072  
P. 5 of 9 YAK

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

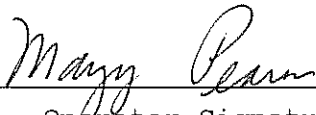
CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-003072  
Location = TOXL                    8164.12.00 06/09  
08/26/2009                            17:10

Test	AC	Time
01 Diagnostic	OK	17:11
02 Room Air	RFI*	17:11
03 Room Air	0.000	17:11

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 11/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Simul Ser No = 000000  
Std Sol No = 00000  
County = 08                            Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

*RFI*

Form 106-I8000

80-003072  
P. 6 of 9 FILE


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003072  
Location = TOXL      8164.12.00 06/09  
08/26/2009      14:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:00
02 Std. Gas	0.081	15:00
03 Room Air	0.000	15:01
04 Std. Gas	0.081	15:01
05 Room Air	0.000	15:01
06 Std. Gas	0.079	15:02
07 Room Air	0.000	15:02

Lot No = 617774  
Cyl No = 42  
Exp Date = 05/28/2011  
County = 08      Oper No. = 107501

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

AC A

Form 106-I8000

80-003072  
P. 7/9 YAK

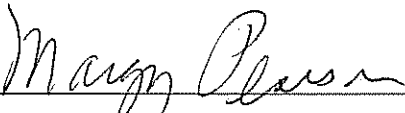
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003072  
Location = TOXL      8164.12.00 06/09  
08/26/2009      15:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:03
02 Std. Gas	0.081	15:04
03 Room Air	0.000	15:04
04 Std. Gas	0.081	15:05
05 Room Air	0.000	15:05
06 Std. Gas	0.081	15:05
07 Room Air	0.000	15:06

Lot No = 617774  
Cyl No = 42  
Exp Date = 05/28/2011  
County = 08      Oper No. = 107501

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-003072  
P. 8 of 9 AL




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003072  
Location = TOXL      8164.12.00 06/09  
08/26/2009      15:06

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:07
02 Std. Gas	0.081	15:07
03 Room Air	0.000	15:08
04 Std. Gas	0.081	15:08
05 Room Air	0.000	15:09
06 Std. Gas	0.081	15:09
07 Room Air	0.000	15:10

Lot No = 617774  
Cyl No = 42  
Exp Date = 05/28/2011  
County = 08      Oper No. = 107501

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-003072  
P. 9 of 9 JCM