

✓ CML

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004193

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓

- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5790 Lot No. — Rep AC 0.021
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5113 Lot No. — Rep AC 0.297
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓
Sim SN DR5115 Lot No. — Rep AC 0.10 AC + Acetone
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓

K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

map

Lot No. 617774 Cyl No. 24 Exp. Date 5/28/04 2011

Test 1 <u>0.080</u>	Test 4 <u>0.079</u>	Test 7 <u>0.080</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.080</u>
Test 3 <u>0.079</u>	Test 6 <u>0.079</u>	Test 9 <u>0.080</u>
Average <u>0.079</u>		

L. Remarks/Maintenance record: Annual Inspection

26 August 2009
Date

Maryn Pearson
Field Inspector's Signature

04 Sept 2009
Date

Deb Kasher
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.12.00 06/09
08/25/2009 16:46

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes

of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 617774
Standard Cyl #? 24
Standard Expiration? 05/28/2011
Oper No? 107501

Flow Cal. Date: 06/04/2009
Slope 692
Intercept -611138

IR Calibration Date: 06/03/2009
 3um 9um

0th Coef(*100):	-12132	-15777
1st Coef(*100):	261160	129907
2nd Coef(*100):	1958	971
H2O adj(mg/l*10k):	413	335

***** Printer Test End *****



Operator Signature
MARGY PEARSON

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.12.00 06/09
08/25/2009 17:50

WET CAL CHECK


Test	AC	Time
01 Room Air	0.000	17:51
02 Std. Sol.	0.017	17:52
03 Room Air	0.000	17:53
04 Std. Sol.	0.018	17:53
05 Room Air	0.000	17:54
06 Std. Sol.	0.019	17:55
07 Room Air	0.000	17:55
08 Sim Temp = 34.0°C		

Simul Ser No = DR00000

Std Sol No = 0000

County = 08

Oper No. = 107501

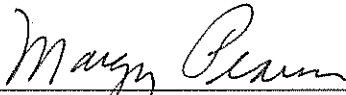

Operator Signature
MARGY PEARSON

Remarks:

Low AC

Form 106-I8000

80-004193
P. 3 of 9 YAH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.12.00 06/09
08/25/2009 18:03

WET CAL CHECK


Test	AC	Time
01 Room Air	0.000	18:04
02 Std. Sol.	0.304	18:04
03 Room Air	0.000	18:05
04 Std. Sol.	0.310	18:06
05 Room Air	0.000	18:06
06 Std. Sol.	0.308	18:07
07 Room Air	0.000	18:07
08 Sim Temp = 34.0°C		

Simul Ser No = DR00000

Std Sol No = 0000

County = 08

Oper No. = 107501

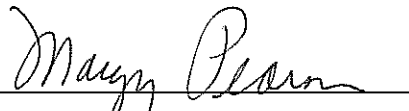

Operator Signature
MARGY PEARSON

Remarks:

Linearity

Form 106-I8000

80-004193
p. 4 of 9 KM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.12.00 06/09
08/26/2009 07:32

Test	AC	Time
01 Room Air	0.000	07:33
02 *Subject Test	INT*	07:34
03 Room Air	0.000	07:35

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR00000
Std Sol No = 0000
County = 08 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

Interferent

Form 106-I8000

80-004193
P. 5 of 9 PM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.12.00 06/09
08/25/2009 17:19


Test	AC	Time
01 Diagnostic	OK	17:20
02 Room Air	RFI*	17:21
03 Room Air	0.000	17:21

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 617774
Cyl No = 24

Expiration Date = 05/28/2011
County = 08 Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

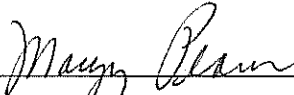

Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-18000

80-004193
P. 6 of 9 1/11

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.12.00 06/09
08/25/2009 16:58

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:59
02 Std. Gas	0.080	16:59
03 Room Air	0.000	16:59
04 Std. Gas	0.079	17:00
05 Room Air	0.000	17:00
06 Std. Gas	0.079	17:01
07 Room Air	0.000	17:01


Lot No = 617774

Cyl No = 24

Exp Date = 05/28/2011

County = 08

Oper No. = 107501

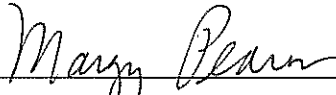

Operator Signature
MARGY PEARSON

Remarks:

Form 106-I8000

ACA

80-004193
p. 7-9 FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.12.00 06/09
08/25/2009 17:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:04
02 Std. Gas	0.079	17:05
03 Room Air	0.000	17:05
04 Std. Gas	0.079	17:06
05 Room Air	0.000	17:06
06 Std. Gas	0.079	17:06
07 Room Air	0.000	17:07


Lot No = 617774

Cyl No = 24

Exp Date = 05/28/2011

County = 08

Oper No. = 107501

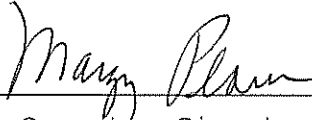

Operator Signature
MARGY PEARSON

Remarks:

Form 106-I8000

ACA

80-004193
P. 8 of 9 ftm

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.12.00 06/09
08/25/2009 17:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:09
02 Std. Gas	0.080	17:09
03 Room Air	0.000	17:10
04 Std. Gas	0.080	17:10
05 Room Air	0.000	17:11
06 Std. Gas	0.080	17:11
07 Room Air	0.000	17:11


Lot No = 617774

Cyl No = 24

Exp Date = 05/28/2011

County = 08

Oper No. = 107501

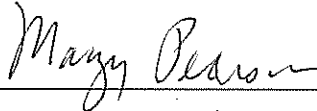

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004193
P. 949 KM