

✓ CML

**INTOXILYZER 5000 G and 8000 INSPECTION**

Serial No.: 80-004206

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR5144 Lot No. --- Rep AC 0.021 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR5187 Lot No. --- Rep AC 0.290 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.) ✓  
Sim SN DR5114 Lot No. --- Rep AC 0.100
- J. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. 585711 Cyl No. 10 Exp. Date 03 Mar 2011  
Test 1 0.080 Test 4 0.079 Test 7 0.079  
Test 2 0.080 Test 5 0.079 Test 8 0.080  
Test 3 0.079 Test 6 0.080 Test 9 0.080  
Average 0.080

L. Remarks/Maintenance record: Annual inspection

26 Aug 09  
Date

Deb Keshu  
Field Inspector's Signature

4 Sept 2009  
Date

Mary Plann  
Reviewed by

80-004206  
p. 6 of 9  
1/22

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000       SN 80-004206  
Location = TOXL               8164.12.00 06/09  
08/26/2009                       16:02

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:           Enabled  
Start Test Sequence:       DABACABA  
Display Prelim Rslt?       Yes  
Display Third Digit?       Yes  
Inhib Printer(Y/N)?       No  
Display Volume?           No  
Disable On Memfull?       Yes  
# of Print Copies?         1  
Select Std (D/W/I)?       Dry  
Standard Value?           0.080  
Standard Lot #?           585711  
Standard Cyl #?           10  
Standard Expiration?      03/03/2011  
Oper No?                   777777

Flow Cal. Date:           06/10/2009  
Slope                      690  
Intercept                 -645883

IR Calibration Date:      06/10/2009

	3um	9um
0th Coef(*100):	-16171	-25317
1st Coef(*100):	259064	128821
2nd Coef(*100):	2464	1154
H2O adj(mg/l*10k):	521	622

\*\*\*\*\* Printer Test End \*\*\*\*\*

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

Print test

Form 106-I8000

80-004206  
P. 2 of 9  
JAN

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004206  
Location = TOXL      8164.12.00 06/09  
08/26/2009      16:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:24
02 Std. Sol.	0.016	16:25
03 Room Air	0.000	16:25
04 Std. Sol.	0.018	16:26
05 Room Air	0.000	16:27
06 Std. Sol.	0.017	16:27
07 Room Air	0.000	16:28

08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 777777

*Deb Kashur*

Operator Signature  
DEB KASHUR

Remarks:

*low AC*

Form 106-I8000

80-004206  
p. 3 of 9

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004206  
Location = TOXL      8164.12.00 06/09  
08/26/2009      16:17

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:18
02 Std. Sol.	0.279	16:20
03 Room Air	0.000	16:20
04 Std. Sol.	0.288	16:21
05 Room Air	0.000	16:21
06 Std. Sol.	0.292	16:22
07 Room Air	0.000	16:23

08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*Linearity test*

Form 106-I8000

80-004206  
P. 4 of 9 JAH

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004206  
Location = TOXL                8164.12.00 06/09  
08/26/2009                        16:06

Test	AC	Time
01 Room Air	0.000	16:07
02 *Subject Test	INT*	16:08
03 Room Air	0.000	16:08

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 585711  
Cyl No = 10  
Expiration Date = 03/03/2011  
County = 08                          Oper No. = 777777

*Deb Kashur*

Operator Signature  
DEB KASHUR

Remarks:

*Invalid test DK - Interferent test*

Form 106-I8000

80-004206  
P. Sat 9/26

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004206  
Location = TOXL                    8164.12.00 06/09  
08/26/2009                            16:29

Test	AC	Time
01 Diagnostic	OK	16:30
02 Room Air	RFI*	16:31
03 Room Air	0.000	16:31

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Simul Ser No = DR0000  
Std Sol No = 000  
County = 08                            Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

80-004206  
P. 6 of 9


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004206  
Location = TOXL      8164.12.00 06/09  
08/26/2009      16:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:33
02 Std. Gas	0.080	16:33
03 Room Air	0.000	16:34
04 Std. Gas	0.080	16:34
05 Room Air	0.000	16:34
06 Std. Gas	0.079	16:35
07 Room Air	0.000	16:35

Lot No = 585711  
Cyl No = 10  
Exp Date = 03/03/2011  
County = 08      Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*

Form 106-I8000

*80-004206  
P. 7049 FR*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004206  
Location = TOXL      8164.12.00 06/09  
08/26/2009      16:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:36
02 Std. Gas	0.079	16:36
03 Room Air	0.000	16:37
04 Std. Gas	0.079	16:37
05 Room Air	0.000	16:38
06 Std. Gas	0.080	16:38
07 Room Air	0.000	16:39

Lot No = 585711  
Cyl No = 10  
Exp Date = 03/03/2011  
County = 08      Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*

Form 106-I8000

80-004206  
P. 8 of 9 JK



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004206  
Location = TOXL      8164.12.00 06/09  
08/26/2009      16:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:42
02 Std. Gas	0.079	16:43
03 Room Air	0.000	16:43
04 Std. Gas	0.080	16:43
05 Room Air	0.000	16:44
06 Std. Gas	0.080	16:44
07 Room Air	0.000	16:45

Lot No = 585711  
Cyl No = 10  
Exp Date = 03/03/2011  
County = 08      Oper No. = 777777

*Deb Kashur*

Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*

Form 106-I8000

80-004206  
P. 9 of 9 AM