

OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 Instrument Certification and Maintenance Unit

BrF-005-8

Q. Flow Monitor. Level 3 Func D Sub F. R 175 F 0.000 L/s
 R. Temperature Monitor. Level 3 Func D Sub T
 C 47.0 A 47.0 I 0.0
 B 45.0 V 0.0
 S. Gas Tank Monitor. Level 3 Func D Sub G 546 psi Intox 500 psi Reg
 T. Configure Wet Bath Sim. Level 1 Func S
 U. Zero Alcohol Test Level 1 Func C
 G 10329 Sim SN — Lot 0.000 Rep Value
0.000 AC 0000 AC 0.000 AC
 V. Low AC Test. Level 1 Func C
DR5190 Sim SN — Lot 0.021 Rep Value
0.019 AC 0.020 AC 0.020 AC
 W. Linearity Test. Level 1 Func C
DR5113 Sim SN — Lot 0.297 Rep Value
0.301 AC 0.301 AC 0.300 AC
 X. Interferent Test. Level 1 Func B
 Acetone DR3845 Sim SN — Lot 0.05% Conc
 Display Printout
 Methanol DR3846 Sim SN — Lot 0.10 Conc
 Display Printout
 IPA DR5191 Sim SN — Lot 0.10% Conc
 Display Printout
 Y. Ambient Test Display Printout
 Z. Invalid Sample Test Display Printout
 AA. RFI Detector Test. CMS Mode Display Printout
 BB. Configure Dry Gas Standard. Level 1 Func S
 CC. Stability Test at 0.080AC. Level 3 Func T Sub S
0.0798 Ave. 0.0006 Std Dev 0.81% Rel Std Dev

DD. Maintenance/Repairs _____

27 August 2009
 Date
04 Sept 2009
 Date

Mary Plaso
 Signature
Deb Kashur
 Reviewed by

Intoxilyzer Test Record and Checklist
ND00AG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alconol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.12.00 06/09
08/27/2009 08:00

***** Printer Test *****

12 char/line
abcdefghijkl
ABCDEFGHIJKL

16 char per line
abcdefghijklmnop
ABCDEFGHIJKLMNPO

21 chars per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMNOPQRSTU

24 characters per line
abcdefghijklmnopqrstuwx
ABCDEFGHIJKLMNOPQRSTUWX

32 characters per line
abcdefghijklmnopqrstuwx123456
ABCDEFGHIJKLMNOPQRSTUWXYZ123456

42 characters per line
abcdefghijklmnopqrstuwx1234567890abcdef
ABCDEFGHIJKLMNOPQRSTUWXYZ1234567890ABCDEF

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On MemFull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 585711
Standard Expiration? 107501
Oper No? 107501

Flow Cal. Date: 04/08/2008
Slope 702
Intercept -647610

IR Calibration Date: 06/04/2008
Sun Sun

0th Coef(*100): -5190 -7737
1st Coef(*100): 270580 138779
2nd Coef(*100): 2499 956
H2O adj(ng/l*10K): 377 188

***** Test End *****

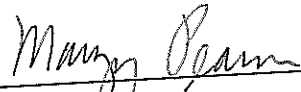
Operator Signature
MARGY PEARSON

Remarks:

Form 106-18000

80-003060
P.3 of 17
AM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.12.00 06/09
08/27/2009 08:02

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 585711
Standard Cyl #? 585711
Standard Expiration? 107501
Oper No? 107501

Flow Cal. Date: 04/08/2008
Slope 702
Intercept -647610

IR Calibration Date: 06/04/2008

	3um	9um
0th Coef(*100):	-5190	-7737
1st Coef(*100):	270580	138779
2nd Coef(*100):	2499	956
H2O adj(mg/l*10k):	377	188

***** Printer Test End *****

Margy Pearson

Operator Signature
MARGY PEARSON

Remarks:

Print Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.12.00 06/09
08/27/2009 08:13

Test	AC	Time
01 Room Air	0.000	08:13
02 *Subject Test	REF*	08:14
03 Room Air	0.000	08:14

*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No =
Cyl No = 585711
Expiration Date =
County = 08 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:



Form 106-I8000

80-003060
P. 5417 PM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.12.00 06/09
08/27/2009 08:16

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 107501

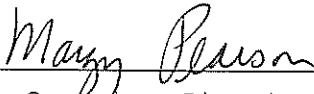

Operator Signature
MARGY PEARSON

Remarks:

DIAG

Form 106-I8000

80-003060
P. 6 of 17 rth

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.12.00 06/09
08/27/2009 11:17

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:17
02 Std. Sol.	0.000	11:19
03 Room Air	0.000	11:20
04 Std. Sol.	0.000	11:20
05 Room Air	0.000	11:21
06 Std. Sol.	0.000	11:21
07 Room Air	0.000	11:22


08 Sim Temp = 34.0°C

Simul Ser No = DR00000

Std Sol No = 0000

County = 08

Oper No. = 107501

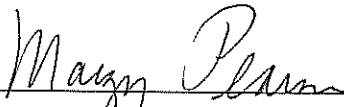

Operator Signature
MARGY PEARSON

Remarks:

6.00 AC Water

Form 106-I8000

80-003060
P. 2 of 17 FL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.12.00 06/09
08/27/2009 10:45

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:46
02 Std. Sol.	0.019	10:47
03 Room Air	0.000	10:47
04 Std. Sol.	0.020	10:48
05 Room Air	0.000	10:49
06 Std. Sol.	0.020	10:49
07 Room Air	0.000	10:50

08 Sim Temp = 34.0°C

Simul Ser No = DR00000
Std Sol No = 0000
County = 08 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

LOW 0.021 AC

Form 106-I8000

80-003060
P. 8 of 17 JCH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.12.00 06/09
08/27/2009 10:57

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:57
02 Std. Sol.	0.301	10:59
03 Room Air	0.000	10:59
04 Std. Sol.	0.301	11:00
05 Room Air	0.000	11:00
06 Std. Sol.	0.300	11:01
07 Room Air	0.000	11:02


08 Sim Temp = 34.0°C

Simul Ser No = DR00000

Std Sol No = 0000

County = 08

Oper No. = 107501

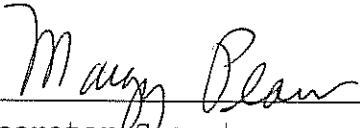
Operator Signature
MARGY PEARSON

Remarks:

Linearity 0.297 AC

Form 106-I8000

80-003060
p.9 of 17 FM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.12.00 06/09
08/27/2009 11:29

Test	AC	Time
01 Room Air	0.000	11:30
02 *Subject Test	INT*	11:30
03 Room Air	0.000	11:31

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR00000
Std Sol No = 0000
County = 08 Oper No. = 107501

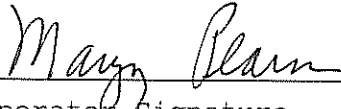

Operator Signature
MARGY PEARSON

Remarks:

Acetone

Form 106-I8000

80-003060
p. 10 of 17
JAN


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.12.00 06/09
08/27/2009 11:32

Test	AC	Time
01 Room Air	0.000	11:32
02 *Subject Test	INT*	11:33
03 Room Air	0.000	11:34

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR00000
Std Sol No = 0000
County = 08 Oper No. = 107501

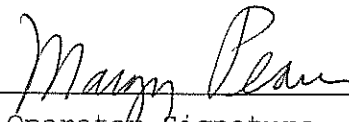

Operator Signature
MARGY PEARSON

Remarks:

Methanol

Form 106-I8000

80-003060
P. 11 of 17 *YH*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.12.00 06/09
08/27/2009 11:26

Test	AC	Time
01 Room Air	0.000	11:26
02 *Subject Test	INT*	11:27
03 Room Air	0.000	11:28

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR00000
Std Sol No = 0000
County = 08 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

IPA

Form 106-I8000

80-003060
P. 12 of 17
YH


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.12.00 06/09
08/27/2009 11:34

Test	AC	Time
01 Room Air	AMB*	11:35
02 Room Air	0.000	11:35

*Invalid Test
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR00000
Std Sol No = 0000
County = 08 Oper No. = 107501

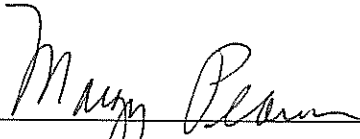

Operator Signature
MARGY PEARSON

Remarks:

Ambient Fail
Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.12.00 06/09
08/27/2009 11:54

Test	AC	Time
01 Room Air	0.000	11:54
02 Invalid Sample	X.XXX	11:54
03 Room Air	0.000	11:55

*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR00000
Std Sol No = 0000
County = 08 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

Invalid Sample

Form 106-I8000

80-003060
P. 14-17 AL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.12.00 06/09
08/27/2009 09:45


Test	AC	Time
01 Diagnostic	OK	09:46
02 Room Air	RFI*	09:46
03 Room Air	0.000	09:46

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 622252
Cyl No = 28
Expiration Date = 05/28/2011
County = 08 Oper No. = 107501

MAP
05/28/2011

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

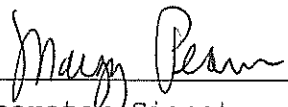

Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-003060
P. 15 of 17 JAK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.12.00 06/09
08/27/2009 08:58

Test	AC	Time
01 Room Air	0.000	09:00
02 Std. Gas	0.079	09:00
03 Room Air	0.000	09:01
04 Std. Gas	0.079	09:01
05 Room Air	0.000	09:02
06 Std. Gas	0.079	09:02
07 Room Air	0.000	09:03
08 Std. Gas	0.080	09:03
09 Room Air	0.000	09:04
10 Std. Gas	0.080	09:04
11 Room Air	0.000	09:05
12 Std. Gas	0.080	09:05
13 Room Air	0.000	09:05
14 Std. Gas	0.079	09:06
15 Room Air	0.000	09:06
16 Std. Gas	0.080	09:07
17 Room Air	0.000	09:07
18 Std. Gas	0.081	09:08
19 Room Air	0.000	09:08
20 Std. Gas	0.080	09:08
21 Room Air	0.000	09:09
22 Std. Gas	0.080	09:09
23 Room Air	0.000	09:10
24 Std. Gas	0.081	09:10
25 Room Air	0.000	09:11
26 Std. Gas	0.079	09:11
27 Room Air	0.000	09:12
28 Std. Gas	0.079	09:12
29 Room Air	0.000	09:12
30 Std. Gas	0.080	09:13
31 Room Air	0.000	09:13
32 Std. Gas	0.079	09:14
33 Room Air	0.000	09:14
34 Std. Gas	0.080	09:15
35 Room Air	0.000	09:15
36 Std. Gas	0.080	09:15
37 Room Air	0.000	09:16
38 Std. Gas	0.080	09:16
39 Room Air	0.000	09:17
40 Std. Gas	0.080	09:17
41 Room Air	0.000	09:18
42 Cal Check Stats:		
43 Average	0.0798	
44 Std Dev	0.0006	
45 Rel Std Dev	0.81%	

80-003060
P. 16417 AK
Stabil
#1 of 2
MAP

Operator Signature
MARGY PEARSON

Remarks:

LOT No 622252
Cyl No 28
Exp Date 5/26/2011
Form 106-I8000

Stat
#2062
MAP

80-003060
8.12.17 JH