

*✓ cmc*

**INTOXILYZER 5000 G and 8000 INSPECTION**

Serial No.: 80-003065

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓  
Sim SN DR5190 Lot No. — Rep AC 0.021
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓  
Sim SN DR5113 Lot No. — Rep AC 0.297
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓  
Sim SN DR5135 Lot No. — Rep AC 0.10 AC + Acetone
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓

K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. 622252 Cyl No. 28 Exp. Date 5/26/2011  
62257 MRP

Test 1 <u>0.080</u>	Test 4 <u>0.082</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.080</u>	Test 6 <u>0.081</u>	Test 9 <u>0.080</u>

Average 0.081

L. Remarks/Maintenance record: Annual Inspection

27 August 2009 Margy Reson  
Date Field Inspector's Signature

04 Sept 2009 Deb Kashner  
Date Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003065  
Location = TOXL      8164.12.00 06/09  
08/27/2009      09:37

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:      Enabled  
Start Test Sequence:      DABACABA  
Display Prelim Rslt?      Yes  
Display Third Digit?      Yes  
Inhib Printer(Y/N)?      No  
Display Volume?      No  
Disable On Memfull?      Yes  
# of Print Copies?      1  
Select Std (D/W/I)?      Dry  
Standard Value?      0.080  
Standard Lot #?      622252  
Standard Cyl #?      28  
Standard Expiration?      05/28/2011  
Oper No?      107501

Flow Cal. Date:      04/08/2008  
Slope      694  
Intercept      -539075

IR Calibration Date:      06/05/2008  
                                    3um                      9um

	3um	9um
0th Coef(*100):	-9383	-9235
1st Coef(*100):	288128	138520
2nd Coef(*100):	957	781
H2O adj(mg/l*10k):	391	239

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
MARGY PEARSON

Remarks:

*Print*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003065  
Location = TOXL      8164.12.00 06/09  
08/27/2009      09:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:33
02 Std. Gas	0.080	09:33
03 Room Air	0.000	09:34
04 Std. Gas	0.081	09:34
05 Room Air	0.000	09:35
06 Std. Gas	0.080	09:35
07 Room Air	0.000	09:36

Lot No = 622252  
Cyl No = 28 *5/26/2011 map*  
Exp Date = 05/28/2011  
County = 08      Oper No. = 107501

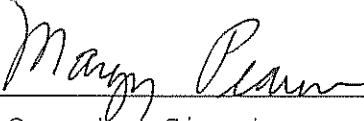

Operator Signature  
MARGY PEARSON

Remarks:

*ACA*

Form 106-I8000

*80-003065  
p. 3 of 9  
FM*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003065  
Location = TOXL      8164.12.00 06/09  
08/27/2009      09:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:38
02 Std. Gas	0.082	09:38
03 Room Air	0.000	09:39
04 Std. Gas	0.081	09:39
05 Room Air	0.000	09:40
06 Std. Gas	0.081	09:40
07 Room Air	0.000	09:41

Lot No = 622252  
Cyl No = 28  
Exp Date = 05/28/2011  
County = 08  
Oper No. = 107501

05/26/2011  
MAP

*Margy Pearson*

Operator Signature  
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-003065  
P. 4 of 9  
KH

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003065  
Location = TOXL      8164.12.00 06/09  
08/27/2009      09:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:43
02 Std. Gas	0.081	09:43
03 Room Air	0.000	09:44
04 Std. Gas	0.081	09:44
05 Room Air	0.000	09:44
06 Std. Gas	0.080	09:45
07 Room Air	0.000	09:45

Lot No = 622252  
Cyl No = 28  
Exp Date = ~~05/28/2011~~  
County = 08  
Oper No. = 107501

5/26/2011  
MAP

Margy Pearson  
Operator Signature  
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-003065  
P. Saf 9/11/11

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003065  
Location = TOXL      8164.12.00 06/09  
08/27/2009      09:47

Test	AC	Time
01 Diagnostic	OK	09:48
02 Room Air	RFI*	09:48
03 Room Air	0.000	09:48

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 11/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 622252  
Cyl No = 28      *5/26/2011 map*  
Expiration Date = *05/28/2011*  
County = 08      Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

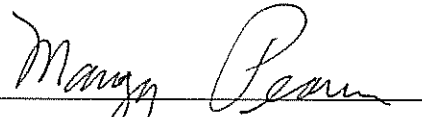

Operator Signature  
MARGY PEARSON

Remarks:

*RFI*

Form 106-I8000

*80-003065  
P. 2 of 9 KM*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003065  
Location = TOXL      8164.12.00 06/09  
08/27/2009      10:20

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:21
02 Std. Sol.	0.290	10:22
03 Room Air	0.000	10:22
04 Std. Sol.	0.297	10:23
05 Room Air	0.000	10:24
06 Std. Sol.	0.296	10:24
07 Room Air	0.000	10:25


08 Sim Temp = 34.0°C

Simul Ser No = DR00000

Std Sol No = 000

County = 08

Oper No. = 107501

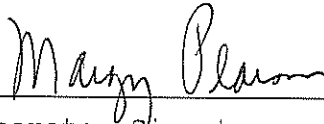

Operator Signature  
MARGY PEARSON

Remarks:

Linearity

Form 106-I8000

80-003065  
P. 7 of 9 FM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003065  
Location = TOXL      8164.12.00 06/09  
08/27/2009      11:05

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:06
02 Std. Sol.	0.019	11:07
03 Room Air	0.000	11:08
04 Std. Sol.	0.019	11:08
05 Room Air	0.000	11:09
06 Std. Sol.	0.020	11:10
07 Room Air	0.000	11:10


08 Sim Temp = 34.0°C

Simul Ser No = DR00000

Std Sol No = 000

County = 08

Oper No. = 107501

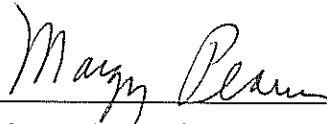

Operator Signature  
MARGY PEARSON

Remarks:

LOW 0.021 AC

Form 106-I8000

80-003065  
P. 8 of 9




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003065  
Location = TOXL      8164.12.00 06/09  
08/27/2009      11:10

Test	AC	Time
01 Room Air	0.000	11:11
02 *Subject Test	INT*	11:12
03 Room Air	0.000	11:12

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 11/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS821456  
Simul Ser No = DR00000  
Std Sol No = 000  
County = 08      Oper No. = 107501

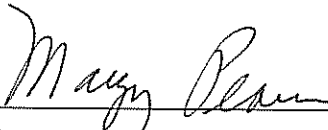  
\_\_\_\_\_  
Operator/Signature  
MARGY PEARSON

Remarks:

*Alc + Acetone*

Form 106-I8000

80-003065  
p. 9 of 9 Jan