

✓ am

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004184

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓
Sim SN DR519 Lot No. - Rep AC 0.02194e
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5113 Lot No. - Rep AC 0.297
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓
Sim SN DR5135 Lot No. - Rep AC 0.10% Ac + Acetone
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. 612623 Cyl No. 9 Exp. Date 05/28/2011
Test 1 0.079 Test 4 0.079 Test 7 0.079
Test 2 0.079 Test 5 0.080 Test 8 0.079
Test 3 0.080 Test 6 0.079 Test 9 0.079
Average 0.079 Ac

L. Remarks/Maintenance record: Annual Inspection
31 August 2009 Marybeth A. Pearson
Date Field Inspector's Signature
04 Sept 2009 Deb Kashner
Date Reviewed by

08/09

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.12.00 06/09
08/28/2009 09:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:36
02 Std. Sol.	0.017	09:36
03 Room Air	0.000	09:37
04 Std. Sol.	0.019	09:38
05 Room Air	0.000	09:38
06 Std. Sol.	0.018	09:39
07 Room Air	0.000	09:40

08 Sim Temp = 34.0°C

Simul Ser No = DR0000
Std Sol No = 0000
County = 08 Oper No. = 107501

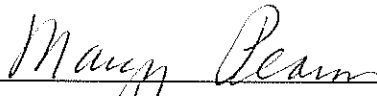

Operator Signature
MARGY PEARSON

Remarks:

LOW AC 0.021 AC

Form 106-I8000

*80-004184
p. 3 of 9 file*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.12.00 06/09
08/28/2009 09:41

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:42
02 Std. Sol.	0.290	09:43
03 Room Air	0.000	09:44
04 Std. Sol.	0.297	09:44
05 Room Air	0.000	09:45
06 Std. Sol.	0.297	09:46
07 Room Air	0.000	09:47


08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 0000

County = 08

Oper No. = 107501

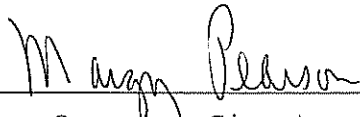

Operator Signature
MARGY PEARSON

Remarks:

Linearity
Form 106-I8000

80-004184
p. 4 of 7 PL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.12.00 06/09
08/31/2009 09:15

Test	AC	Time
01 Room Air	0.000	09:16
02 *Subject Test	INT*	09:18
03 Room Air	0.000	09:19

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR0000
Std Sol No = 0000
County = 08 Oper No. = 107501



Operator/Signature
MARGY PEARSON

Remarks:

Interferant
Form 106-I8000

80-004184
P. 5 of 9 *jev*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.12.00 06/09
08/28/2009 07:09


Test	AC	Time
01 Diagnostic	OK	07:11
02 Room Air	RFI*	07:11
03 Room Air	0.000	07:11

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 612623
Cyl No = 9

Expiration Date = 05/28/2011
County = 08 Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

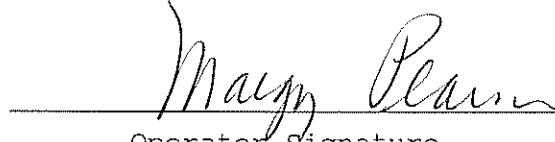

Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-004184
P. 6 of 9 JFL


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.12.00 06/09
08/27/2009 09:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:29
02 Std. Gas	0.079	09:29
03 Room Air	0.000	09:30
04 Std. Gas	0.079	09:30
05 Room Air	0.000	09:31
06 Std. Gas	0.080	09:31
07 Room Air	0.000	09:32

Lot No = 612623
Cyl No = 9
Exp Date = 05/28/2011
County = 08 Oper No. = 107501

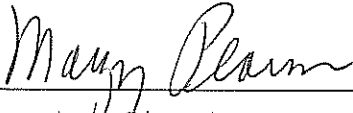

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004184
P. 7 of 7, JKH


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.12.00 06/09
08/27/2009 09:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:34
02 Std. Gas	0.079	09:34
03 Room Air	0.000	09:35
04 Std. Gas	0.080	09:35
05 Room Air	0.000	09:36
06 Std. Gas	0.079	09:36
07 Room Air	0.000	09:37

Lot No = 612623
Cyl No = 9
Exp Date = 05/28/2011
County = 08 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004184
P. 8 of 9

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.12.00 06/09
08/27/2009 09:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:38
02 Std. Gas	0.079	09:38
03 Room Air	0.000	09:39
04 Std. Gas	0.079	09:39
05 Room Air	0.000	09:40
06 Std. Gas	0.079	09:40
07 Room Air	0.000	09:41


Lot No = 612623

Cyl No = 9

Exp Date = 05/28/2011

County = 08

Oper No. = 107501

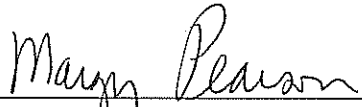

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004184
P. 9 of 9