

✓ *CM*

**INTOXILYZER 5000 G and 8000 INSPECTION**

Serial No.: 80-004203

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓  
Sim SN DR5190 Lot No. — Rep AC 0.021 AC
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓  
Sim SN DR5113 Lot No. — Rep AC 0.1092
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓  
Sim SN DR5135 Lot No. — Rep AC 0.10 AC + Acetone
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. 622252 Cyl No. 28 Exp. Date 5/26/2011

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.082</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>

Average 0.081

L. Remarks/Maintenance record: Annual Inspection

<u>27 August 2009</u> Date	<u>Maryn Plam</u> Field Inspector's Signature
<u>04 Sept 2009</u> Date	<u>Deb Kashner</u> Reviewed by



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004203  
Location = TOXL      8164.12.00 06/09  
08/27/2009      17:03

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:03
02 Std. Sol.	0.020	17:04
03 Room Air	0.000	17:05
04 Std. Sol.	0.019	17:05
05 Room Air	0.000	17:06
06 Std. Sol.	0.019	17:07
07 Room Air	0.000	17:07

08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 107501



Operator/Signature  
MARGY PEARSON

Remarks:

*Low AC 0.021 AC*

Form 106-I8000

80-004203  
P. 3 of 9 JAH

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004203  
Location = TOXL      8164.12.00 06/09  
08/27/2009      17:38

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:39
02 Std. Sol.	0.283	17:40
03 Room Air	0.000	17:40
04 Std. Sol.	0.288	17:41
05 Room Air	0.000	17:42
06 Std. Sol.	0.287	17:42
07 Room Air	0.000	17:43


08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 107501

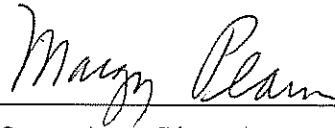

Operator Signature  
MARGY PEARSON

Remarks:

*Linearity*  
~~*interferent*~~ *map*

Form 106-I8000

80-004203  
P. 4/9/12


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004203  
Location = TOXL      8164.12.00 06/09  
08/27/2009      17:26

Test	AC	Time
01 Room Air	0.000	17:27
02 *Subject Test	INT*	17:28
03 Room Air	0.000	17:29

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 11/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS821456  
Simul Ser No = DR0000  
Std Sol No = 000  
County = 08      Oper No. = 107501

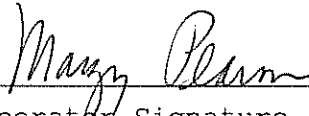

Operator Signature  
MARGY PEARSON

Remarks:

*Interferent*

Form 106-I8000

80-004203  
P. 5/9 PM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004203  
Location = TOXL                    8164.12.00 06/09  
08/27/2009                            16:29

Test	AC	Time
01 Diagnostic	OK	16:30
02 Room Air	0.000	16:31
03 *Subject Test	RFI*	16:31
04 Room Air	0.000	16:31

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 11/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 622252  
Cyl No = 28  
Expiration Date = 05/26/2011  
County = 08                            Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

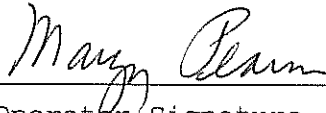

Operator Signature  
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-004203  
P. 6 of 9 PK


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004203  
Location = TOXL      8164.12.00 06/09  
08/27/2009      15:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:42
02 Std. Gas	0.081	15:43
03 Room Air	0.000	15:43
04 Std. Gas	0.082	15:44
05 Room Air	0.000	15:44
06 Std. Gas	0.081	15:44
07 Room Air	0.000	15:45

Lot No = 622252  
Cyl No = 28  
Exp Date = 05/26/2011  
County = 08      Oper No. = 107501

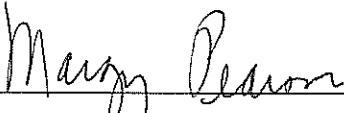  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004203  
P. J. G. R.

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004203  
Location = TOXL      8164.12.00 06/09  
08/27/2009      15:46

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:54
02 Std. Gas	0.081	15:54
03 Room Air	0.000	15:55
04 Std. Gas	0.081	15:55
05 Room Air	0.000	15:56
06 Std. Gas	0.081	15:56
07 Room Air	0.000	15:56

Lot No = 622252  
Cyl No = 28  
Exp Date = 05/26/2011  
County = 08      Oper No. = 107501



Operator Signature  
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004203  
R. J. J. 9/1/09



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004203  
Location = TOXL      8164.12.00 06/09  
08/27/2009      15:58

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:59
02 Std. Gas	0.081	15:59
03 Room Air	0.000	16:00
04 Std. Gas	0.081	16:00
05 Room Air	0.000	16:00
06 Std. Gas	0.081	16:01
07 Room Air	0.000	16:01

Lot No = 622252

Cyl No = 28

Exp Date = 05/26/2011

County = 08

Oper No. = 107501



Operator Signature  
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004203  
P. 9 of 9 VM