

✓ CML

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004188

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5190 Lot No. — Rep AC 0.021
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5113 Lot No. — Rep AC 0.297
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓
Sim SN DR5135 Lot No. — Rep AC 0.10AC + Acetone
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. 617774 Cyl No. 12 Exp. Date 5/28/2011

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.082</u>
Test 2 <u>0.082</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.082</u>

Average 0.081

L. Remarks/Maintenance record: Annual Inspection

28 August 2009 Mary Pearson
Date Field Inspector's Signature

04 Sept 2009 Deb Kasher
Date Reviewed by

08/09

80-004188
p. 1 of 9

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.12.00 06/09
08/28/2009 07:23

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? Yes
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 617774
Standard Cyl #? 12
Standard Expiration? 05/28/2011
Oper No? 107501


Flow Cal. Date: 06/04/2009
Slope 689
Intercept -630047

IR Calibration Date: 06/03/2009

	3um	9um
--	-----	-----

0th Coef(*100):	-23767	-23243
1st Coef(*100):	280491	137567
2nd Coef(*100):	1648	933
H2O adj(mg/l*10k):	733	647

***** Printer Test End *****

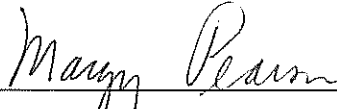

Operator Signature
MARGY PEARSON

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.12.00 06/09
08/28/2009 07:26

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:27
02 Std. Sol.	0.018	07:28
03 Room Air	0.000	07:28
04 Std. Sol.	0.020	07:29
05 Room Air	0.000	07:30
06 Std. Sol.	0.019	07:30
07 Room Air	0.000	07:31


08 Sim Temp = 34.0°C

Simul Ser No = DR000

Std Sol No = 0000

County = 08

Oper No. = 107501

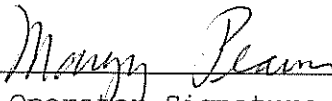

Operator Signature
MARGY PEARSON

Remarks:

LOW 0.021AC

Form 106-I8000

80-004188
P. 3 of 9 FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.12.00 06/09
08/28/2009 08:03

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:04
02 Std. Sol.	0.294	08:05
03 Room Air	0.000	08:06
04 Std. Sol.	0.296	08:06
05 Room Air	0.000	08:07
06 Std. Sol.	0.295	08:08
07 Room Air	0.000	08:08


08 Sim Temp = 34.0°C

Simul Ser No = DR000

Std Sol No = 0000

County = 08

Oper No. = 107501

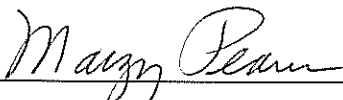

Operator Signature
MARGY PEARSON

Remarks:

Liquority

Form 106-I8000

80-004188
p. 4 of 9 JEM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.12.00 06/09
08/28/2009 08:09

Test	AC	Time
01 Room Air	0.000	08:10
02 *Subject Test	INT*	08:11
03 Room Air	0.000	08:11

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR000
Std Sol No = 0000
County = 08 Oper No. = 107501

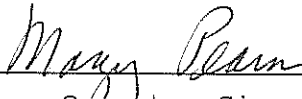

Operator Signature
MARGY PEARSON

Remarks:

Interferent

Form 106-18000

80-004188
P. 5 of 9 PM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.12.00 06/09
08/27/2009 16:48

Test	AC	Time
01 Diagnostic	OK	16:49
02 Room Air	0.000	16:50
03 *Subject Test	RFI*	16:50
04 Room Air	0.000	16:50

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

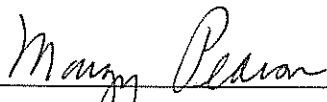

Operator/Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-004188
P. 6 of 9 FR

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.12.00 06/09
08/27/2009 16:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:27
02 Std. Gas	0.081	16:27
03 Room Air	0.000	16:28
04 Std. Gas	0.082	16:28
05 Room Air	0.000	16:29
06 Std. Gas	0.081	16:29
07 Room Air	0.000	16:29


Lot No = 617774

Cyl No = 12

Exp Date = 05/28/2011

County = 08

Oper No. = 107501

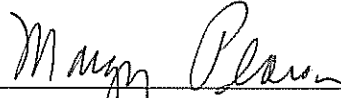

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004188
P. 7 of 9 VM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.12.00 06/09
08/28/2009 07:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:08
02 Std. Gas	0.081	07:09
03 Room Air	0.000	07:09
04 Std. Gas	0.081	07:09
05 Room Air	0.000	07:10
06 Std. Gas	0.081	07:10
07 Room Air	0.000	07:11

Lot No = 617774
Cyl No = 12
Exp Date = 05/28/2011
County = 08 Oper No. = 107501

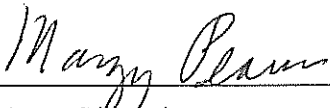

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004188
P. P. 9 KAL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.12.00 06/09
08/28/2009 07:13

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:13
02 Std. Gas	0.082	07:14
03 Room Air	0.000	07:14
04 Std. Gas	0.081	07:14
05 Room Air	0.000	07:15
06 Std. Gas	0.082	07:15
07 Room Air	0.000	07:16


Lot No = 617774

Cyl No = 12

Exp Date = 05/28/2011

County = 08

Oper No. = 107501

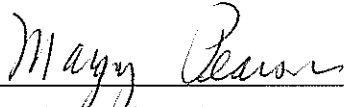

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-18000

80-004188
P. 9 of 9 VJM