

✓ CML

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004194

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓

- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5144 Lot No. — Rep AC 0.021
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5187 Lot No. — Rep AC 0.290
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓
Sim SN DR5114 Lot No. — Rep AC 0.100
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

| | | |
|-----------------------|---------------------|--|
| Lot No. <u>585711</u> | Cyl No. <u>10</u> | Exp. Date <u>03 Mar 2011</u> 2010 |
| Test 1 <u>0.079</u> | Test 4 <u>0.080</u> | Test 7 <u>0.080</u> |
| Test 2 <u>0.079</u> | Test 5 <u>0.080</u> | Test 8 <u>0.079</u> |
| Test 3 <u>0.078</u> | Test 6 <u>0.080</u> | Test 9 <u>0.080</u> |
| Average <u>0.079</u> | | |

L. Remarks/Maintenance record: annual inspection

28 Aug 2009
Date

Deb Kashner
Field Inspector's Signature

4 Sept 2009
Date

Mary Plerson
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004194
Location = TOXL 8164.12.00 06/09
08/27/2009 09:40

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No

of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 585711
Standard Cyl #? 10
Standard Expiration? 03/03/2011
Oper No? 777777

Flow Cal. Date: 06/23/2009
Slope 681
Intercept -622430

IR Calibration Date: 06/23/2009

| | 3um | 9um |
|--------------------|--------|--------|
| 0th Coef(*100): | -16880 | -23747 |
| 1st Coef(*100): | 267556 | 131411 |
| 2nd Coef(*100): | 1978 | 1220 |
| H2O adj(mg/l*10k): | 440 | 575 |

***** Printer Test End *****

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Print test

Form 106-I8000

80-004194
p. 2 of 9 RW

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004194
Location = TOXL 8164.12.00 06/09
08/28/2009 10:56

WET CAL CHECK

| Test | AC | Time |
|----------------------|-------|-------|
| 01 Room Air | 0.000 | 10:56 |
| 02 Std. Sol. | 0.017 | 10:57 |
| 03 Room Air | 0.000 | 10:58 |
| 04 Std. Sol. | 0.017 | 10:58 |
| 05 Room Air | 0.000 | 10:59 |
| 06 Std. Sol. | 0.017 | 11:00 |
| 07 Room Air | 0.000 | 11:00 |
| 08 Sim Temp = 34.0°C | | |

Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Low AC

Form 106-I8000

80-004194
P. 3 of 9

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004194
Location = TOXL 8164.12.00 06/09
08/27/2009 14:25

WET CAL CHECK

| Test | AC | Time |
|----------------------|-------|-------|
| 01 Room Air | 0.000 | 14:25 |
| 02 Std. Sol. | 0.295 | 14:26 |
| 03 Room Air | 0.000 | 14:27 |
| 04 Std. Sol. | 0.297 | 14:28 |
| 05 Room Air | 0.000 | 14:28 |
| 06 Std. Sol. | 0.297 | 14:29 |
| 07 Room Air | 0.000 | 14:29 |
| 08 Sim Temp = 34.0°C | | |

Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Linearity test

Form 106-I8000

80-004194
P. 4 of 9 AM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004194
Location = TOXL 8164.12.00 06/09
08/27/2009 13:40

| Test | AC | Time |
|------------------|-------|-------|
| 01 Room Air | 0.000 | 13:41 |
| 02 *Subject Test | INT* | 13:42 |
| 03 Room Air | 0.000 | 13:43 |

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent check

Form 106-I8000

80-004194
P. 5 of 9 KM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004194
Location = TOXL 8164.12.00 06/09
08/27/2009 13:55

| Test | AC | Time |
|---------------|-------|-------|
| 01 Diagnostic | OK | 13:56 |
| 02 Room Air | RFI* | 13:56 |
| 03 Room Air | 0.000 | 13:56 |

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

80-004194
P. 6 of 9 KAL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004194
Location = TOXL 8164.12.00 06/09
08/27/2009 09:26

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 09:27 |
| 02 Std. Gas | 0.079 | 09:27 |
| 03 Room Air | 0.000 | 09:28 |
| 04 Std. Gas | 0.079 | 09:28 |
| 05 Room Air | 0.000 | 09:29 |
| 06 Std. Gas | 0.078 | 09:29 |
| 07 Room Air | 0.000 | 09:30 |

Lot No = 585711

Cyl No = 10

Exp Date = 03/03/2011

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-004194
P. 7 of 9 KM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004194
Location = TOXL 8164.12.00 06/09
08/27/2009 09:30

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 09:31 |
| 02 Std. Gas | 0.080 | 09:31 |
| 03 Room Air | 0.000 | 09:32 |
| 04 Std. Gas | 0.080 | 09:32 |
| 05 Room Air | 0.000 | 09:32 |
| 06 Std. Gas | 0.080 | 09:33 |
| 07 Room Air | 0.000 | 09:33 |

Lot No = 585711

Cyl No = 10

Exp Date = 03/03/2011

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-004194
P. 8/9 AL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004194
Location = TOXL 8164.12.00 06/09
08/27/2009 09:34

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 09:35 |
| 02 Std. Gas | 0.080 | 09:35 |
| 03 Room Air | 0.000 | 09:36 |
| 04 Std. Gas | 0.079 | 09:36 |
| 05 Room Air | 0.000 | 09:37 |
| 06 Std. Gas | 0.080 | 09:37 |
| 07 Room Air | 0.000 | 09:38 |

Lot No = 585711

Cyl No = 10

Exp Date = 03/03/2011

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-004194
P. 9 of 9