

✓ cm

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-04201

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5190 Lot No. — Rep AC 0.021
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5113 Lot No. — Rep AC 0.297
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓
Sim SN DR5135 Lot No. — Rep AC 0.10 AC + Acetone
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. <u>617774</u>	Cyl No. <u>33</u>	Exp. Date <u>5/28/2011</u>
Test 1 <u>0.081</u>	Test 4 <u>0.082</u>	Test 7 <u>0.081</u>
Test 2 <u>0.082</u>	Test 5 <u>0.081</u>	Test 8 <u>0.082</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.080</u>
Average <u>0.081</u>		

L. Remarks/Maintenance record: Annual Inspection

28 August 2009 Mary Adams
Date Field Inspector's Signature

04 Sept 2009 Deb Kashner
Date Reviewed by

80-004201
P149 FL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = TOXL 8164.12.00 06/09
08/28/2009 09:25

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:26
02 Std. Sol.	0.015	09:27
03 Room Air	0.000	09:28
04 Std. Sol.	0.017	09:28
05 Room Air	0.000	09:29
06 Std. Sol.	0.018	09:30
07 Room Air	0.000	09:30


08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 0000

County = 08

Oper No. = 107501

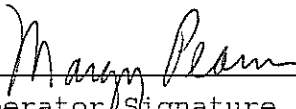

Operator Signature
MARGY PEARSON

Remarks:

LOW 0.021 AC

Form 106-I8000

80-004201
P. 3 of 7 FA

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004201
Location = TOXL 8164.12.00 06/09
08/28/2009 08:58

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:58
02 Std. Sol.	0.292	08:59
03 Room Air	0.000	09:00
04 Std. Sol.	0.299	09:00
05 Room Air	0.000	09:01
06 Std. Sol.	0.297	09:02
07 Room Air	0.000	09:02


08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 107501

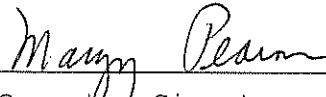

Operator Signature
MARGY PEARSON

Remarks:

Linearity

Form 106-I8000

80-004201
P. 4 of 9 fm

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004201
Location = TOXL 8164.12.00 06/09
08/28/2009 10:22

Test	AC	Time
01 Room Air	0.000	10:22
02 *Subject Test	INT*	10:23
03 Room Air	0.000	10:24

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

Alc + Acetone

Form 106-I8000

80-004201
P. 5-17-9 AL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004201
Location = TOXL 8164.12.00 06/09
08/28/2009 07:09

Test	AC	Time
01 Diagnostic	OK	07:10
02 Room Air	RFI*	07:11
03 Room Air	0.000	07:11

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 617774
Cyl No = 33
Expiration Date = 05/28/2011
County = 08 Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

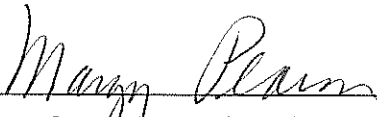

Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-004201
P. 6 of 9 pm


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004201
Location = TOXL 8164.12.00 06/09
08/27/2009 09:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:29
02 Std. Gas	0.082	09:29
03 Room Air	0.000	09:30
04 Std. Gas	0.081	09:30
05 Room Air	0.000	09:31
06 Std. Gas	0.081	09:31
07 Room Air	0.000	09:32

Lot No = 617774
Cyl No = 33
Exp Date = 05/28/2011
County = 08 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004201
P. 2 of 9 PL


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004201
Location = TOXL 8164.12.00 06/09
08/27/2009 09:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:22
02 Std. Gas	0.081	09:22
03 Room Air	0.000	09:23
04 Std. Gas	0.082	09:23
05 Room Air	0.000	09:24
06 Std. Gas	0.081	09:24
07 Room Air	0.000	09:25

Lot No = 617774
Cyl No = 33
Exp Date = 05/28/2011
County = 08 Oper No. = 107501

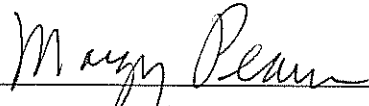

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004201
P-849 PM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004201
Location = TOXL 8164.12.00 06/09
08/27/2009 09:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:38
02 Std. Gas	0.081	09:38
03 Room Air	0.000	09:39
04 Std. Gas	0.082	09:39
05 Room Air	0.000	09:39
06 Std. Gas	0.080	09:40
07 Room Air	0.000	09:40


Lot No = 617774

Cyl No = 33

Exp Date = 05/28/2011

County = 08

Oper No. = 107501

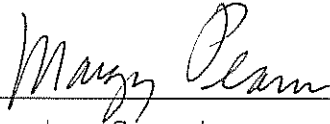

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-18000

80-004201
P. 9 of 9 File