

✓ CML

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004204

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5190 Lot No. — Rep AC 0.021
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5113 Lot No. — Rep AC 0.297
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓
Sim SN DR5135 Lot No. — Rep AC 0.10 AC + Acetone
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. 617774 Cyl No. 24 Exp. Date 5/28/2011
Test 1 0.081 Test 4 0.081 Test 7 0.081
Test 2 0.082 Test 5 0.081 Test 8 0.081
Test 3 0.081 Test 6 0.082 Test 9 0.082
Average 0.081

L. Remarks/Maintenance record: Annual Inspection
31 August 2009 Margaret A. Rosen
Date Field Inspector's Signature
04 Sept 2009 Deb Kashner
Date Reviewed by

08/09

80-004204
Plat. 7 gm

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = TOXL 8164.12.00 06/09
08/31/2009 08:38

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? Yes
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 12345678OP
Standard Cyl #? QWERTU
Standard Expiration? 09/09/2011
Oper No? 107501

Flow Cal. Date: 06/04/2009
Slope 702
Intercept -590070

IR Calibration Date: 06/03/2009
 3um 9um

0th Coef(*100): -8491 -20200
1st Coef(*100): 254748 127473
2nd Coef(*100): 2981 1113
H2O adj(mg/l*10k): 595 541

***** Printer Test End *****

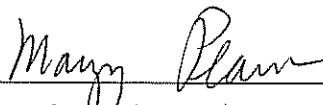

Operator Signature
MARGY PEARSON

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = TOXL 8164.12.00 06/09
08/31/2009 09:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:35
02 Std. Sol.	0.018	09:37
03 Room Air	0.000	09:37
04 Std. Sol.	0.019	09:38
05 Room Air	0.000	09:38
06 Std. Sol.	0.019	09:39
07 Room Air	0.000	09:40


08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 107501

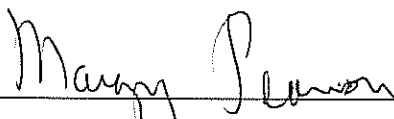

Operator Signature
MARGY PEARSON

Remarks:

LOW 0.021 AC

Form 106-I8000

80-004204
P. 3 of 9 ML

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = TOXL 8164.12.00 06/09
08/31/2009 10:13

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:13
02 Std. Sol.	0.287	10:15
03 Room Air	0.000	10:15
04 Std. Sol.	0.298	10:16
05 Room Air	0.000	10:17
06 Std. Sol.	0.299	10:17
07 Room Air	0.000	10:18

08 Sim Temp = 34.0°C

Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

Liberty
Form 106-I8000

80-004204
Q. 4-7 PM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = TOXL 8164.12.00 06/09
08/31/2009 10:02

Test	AC	Time
01 Room Air	0.000	10:03
02 *Subject Test	INT*	10:04
03 Room Air	0.000	10:04

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 107501

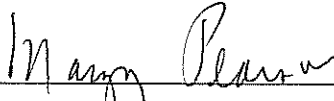

Operator Signature
MARGY PEARSON

Remarks:

Alc + Acetone

Form 106-I8000

80-004204
P. 5 of 9 FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = TOXL 8164.12.00 06/09
08/31/2009 10:38

Test	AC	Time
01 Diagnostic	OK	10:39
02 Room Air	RFI*	10:39
03 Room Air	0.000	10:39

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-004204
P. 6 of 9 FR


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = TOXL 8164.12.00 06/09
08/31/2009 09:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:22
02 Std. Gas	0.081	09:23
03 Room Air	0.000	09:23
04 Std. Gas	0.082	09:24
05 Room Air	0.000	09:24
06 Std. Gas	0.081	09:25
07 Room Air	0.000	09:25

Lot No = 617774
Cyl No = 24
Exp Date = 05/28/2011
County = 08 Oper No. = 107501

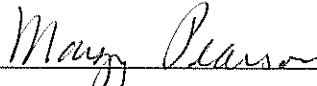

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004204
P. 7/9 FM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = TOXL 8164.12.00 06/09
08/31/2009 09:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:19
02 Std. Gas	0.081	09:19
03 Room Air	0.000	09:19
04 Std. Gas	0.081	09:20
05 Room Air	0.000	09:20
06 Std. Gas	0.082	09:21
07 Room Air	0.000	09:21

Lot No = 617774
Cyl No = 24
Exp Date = 05/28/2011
County = 08 Oper No. = 107501

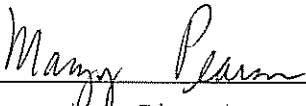

Operator Signature
MARGY PEARSON

Remarks: *ACA*

Form 106-I8000

80-004204
P. P. P. P. P.


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = TOXL 8164.12.00 06/09
08/31/2009 09:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:11
02 Std. Gas	0.081	09:12
03 Room Air	0.000	09:12
04 Std. Gas	0.081	09:13
05 Room Air	0.000	09:13
06 Std. Gas	0.082	09:13
07 Room Air	0.000	09:14

Lot No = 617774
Cyl No = 24
Exp Date = 05/28/2011
County = 08 Oper No. = 107501

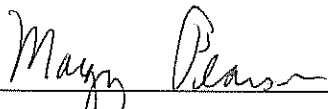

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004204
P. 9 of 9 PM