

✓ CML

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004205

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5190 Lot No. — Rep AC 0.021
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5113 Lot No. — Rep AC 0.297
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓
Sim SN DR5135 Lot No. — Rep AC 0.10AC + ACETONE
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. 612623 Cyl No. 10 Exp. Date 5/28/2011
Test 1 0.081 Test 4 0.080 Test 7 0.081
Test 2 0.081 Test 5 0.080 Test 8 0.081
Test 3 0.081 Test 6 0.081 Test 9 0.080
Average 0.081

L. Remarks/Maintenance record: Annual Inspection
31 August 2009 Mary Blum
Date Field Inspector's Signature
04 Sept 2009 Deb Kashur
Date Reviewed by

80-004205
p. 1 of 9
FK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.12.00 06/09
08/28/2009 07:36

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? Yes
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 12345R67
Standard Cyl #? 1Q23E4
Standard Expiration? 11/11/2011
Oper No? 107501

Flow Cal. Date: 06/04/2009
Slope 704
Intercept -610062

IR Calibration Date: 06/03/2009
3um 9um

	3um	9um
0th Coef(*100):	-16944	-29673
1st Coef(*100):	265939	132317
2nd Coef(*100):	1696	900
H2O adj(mg/l*10k):	490	583

***** Printer Test End *****

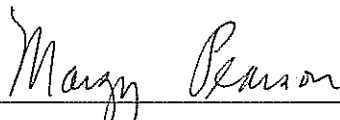

Operator Signature
MARGY PEARSON

Remarks:



Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.12.00 06/09
08/31/2009 17:03

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:04
02 Std. Sol.	0.017	17:05
03 Room Air	0.000	17:06
04 Std. Sol.	0.018	17:07
05 Room Air	0.000	17:07
06 Std. Sol.	0.018	17:08
07 Room Air	0.000	17:08


08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 107501

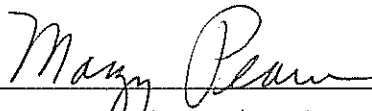

Operator Signature
MARGY PEARSON

Remarks:

LOW 0.021 AC

Form 106-I8000

80-004205
P. 3/9 PM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.12.00 06/09
08/31/2009 09:58

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:58
02 Std. Sol.	0.290	10:00
03 Room Air	0.000	10:00
04 Std. Sol.	0.299	10:01
05 Room Air	0.000	10:01
06 Std. Sol.	0.298	10:02
07 Room Air	0.000	10:03


08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

Linearity

Form 106-I8000

*80-004205
P 4/9 FL*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.12.00 06/09
08/31/2009 10:04

Test	AC	Time
01 Room Air	0.000	10:05
02 *Subject Test	INT*	10:05
03 Room Air	0.000	10:06

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 107501

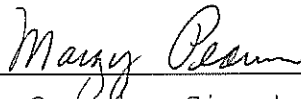

Operator Signature
MARGY PEARSON

Remarks:

Alc + Acetone

Form 106-I8000

80-004205
P. 5/9/09

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.12.00 06/09
08/31/2009 09:44

Test	AC	Time
01 Diagnostic	OK	09:45
02 Room Air	RFI*	09:45
03 Room Air	0.000	09:46

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

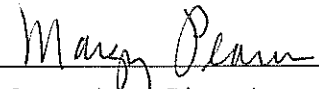

Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-004205
P. 6 of 7 FM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.12.00 06/09
08/31/2009 09:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:14
02 Std. Gas	0.081	09:15
03 Room Air	0.000	09:15
04 Std. Gas	0.081	09:16
05 Room Air	0.000	09:16
06 Std. Gas	0.081	09:16
07 Room Air	0.000	09:17

Lot No = 612623
Cyl No = 10
Exp Date = 05/28/2011
County = 08 Oper No. = 107501

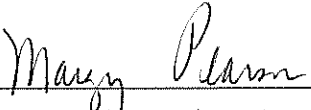

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-18000

80-004205
P. 2 of 9 JDR


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.12.00 06/09
08/31/2009 09:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:19
02 Std. Gas	0.080	09:19
03 Room Air	0.000	09:20
04 Std. Gas	0.080	09:20
05 Room Air	0.000	09:20
06 Std. Gas	0.081	09:21
07 Room Air	0.000	09:21

Lot No = 612623
Cyl No = 10
Exp Date = 05/28/2011
County = 08 Oper No. = 107501

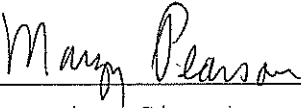

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004205
P. 8 of 9 FH


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.12.00 06/09
08/31/2009 09:23

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:23
02 Std. Gas	0.081	09:24
03 Room Air	0.000	09:24
04 Std. Gas	0.081	09:24
05 Room Air	0.000	09:25
06 Std. Gas	0.080	09:25
07 Room Air	0.000	09:26

Lot No = 612623
Cyl No = 10
Exp Date = 05/28/2011
County = 08 Oper No. = 107501

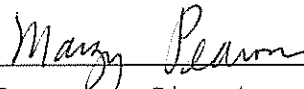

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004205
P. 9 of 9 PM