

✓ cmc

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-003061

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5144 Lot No. — Rep AC 0-021 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5187 Lot No. — Rep AC 0-290 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR5114 Lot No. — Rep AC 0-100 ✓
- J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓

K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. <u>617774</u>	Cyl No. <u>48</u>	Exp. Date <u>28 May 2011</u>
Test 1 <u>0.080</u>	Test 4 <u>0.081</u>	Test 7 <u>0.082</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.080</u>	Test 6 <u>0.082</u>	Test 9 <u>0.081</u>
Average <u>0.081</u>		

L. Remarks/Maintenance record: Annual inspection

01 Sept 09
Date

Deb Kasheer
Field Inspector's Signature

4 Sept 09
Date

Mary Plummer
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.12.00 06/09
09/01/2009 10:51

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 617774
Standard Cyl #? 38
Standard Expiration? 05/28/2011
Oper No? 777777

Flow Cal. Date: 06/15/2009
Slope 737
Intercept -507305

IR Calibration Date: 06/04/2008

	3um	9um
0th Coef(*100):	-10373	-9786
1st Coef(*100):	269813	141172
2nd Coef(*100):	1454	683
H2O adj(mg/l*10k):	474	231

***** Printer Test End *****

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Print test

Form 106-18000

80-003061
P. 2 of 7
FK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.12.00 06/09
09/01/2009 13:42

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:42
02 Std. Sol.	0.019	13:44
03 Room Air	0.000	13:44
04 Std. Sol.	0.020	13:45
05 Room Air	0.000	13:45
06 Std. Sol.	0.021	13:46
07 Room Air	0.000	13:47

08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Low AC

Form 106-I8000

*80-003061
p. 3 of 9
JFW*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.12.00 06/09
09/01/2009 11:30

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:31
02 Std. Sol.	0.294	11:32
03 Room Air	0.000	11:33
04 Std. Sol.	0.296	11:34
05 Room Air	0.000	11:34
06 Std. Sol.	0.296	11:35
07 Room Air	0.000	11:36

08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Linearity test

Form 106-I8000

80-003061
P. 4 of 9
AK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.12.00 06/09
09/01/2009 14:03

Test	AC	Time
01 Room Air	0.000	14:04
02 *Subject Test	INT*	14:04
03 Room Air	0.000	14:05

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent Test

Form 106-18000

80-003061
P. 5 of 9
FK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.12.00 06/09
09/01/2009 11:02

Test	AC	Time
01 Diagnostic	OK	11:03
02 Room Air	RFI*	11:03
03 Room Air	0.000	11:04

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

RFI Test

Form 106-I8000

80-003061
P. 6 of 9 pm

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.12.00 06/09
09/01/2009 09:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:53
02 Std. Gas	0.080	09:54
03 Room Air	0.000	09:54
04 Std. Gas	0.081	09:55
05 Room Air	0.000	09:55
06 Std. Gas	0.080	09:55
07 Room Air	0.000	09:56

Lot No = 617774
Cyl No = 38
Exp Date = 05/28/2011
County = 08 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-18000

80-003061
P. 7 of 9 *AK*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.12.00 06/09
09/01/2009 10:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:39
02 Std. Gas	0.081	10:39
03 Room Air	0.000	10:40
04 Std. Gas	0.081	10:40
05 Room Air	0.000	10:41
06 Std. Gas	0.082	10:41
07 Room Air	0.000	10:42

Lot No = 617774

Cyl No = 38

Exp Date = 05/28/2011

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

OK Calibration test

Form 106-18000

80-003061
P. 8 of 9 JEM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.12.00 06/09
09/01/2009 10:46

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:47
02 Std. Gas	0.082	10:47
03 Room Air	0.000	10:48
04 Std. Gas	0.081	10:48
05 Room Air	0.000	10:49
06 Std. Gas	0.081	10:49
07 Room Air	0.000	10:50

Lot No = 617774
Cyl No = 38
Exp Date = 05/28/2011
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003061
P. 9 of 9
JK