

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 Instrument Certification and Maintenance Unit

BrF-005-8

Q. Flow Monitor. Level 3 Func D Sub F. R 177 F 0.000 L/s
 R. Temperature Monitor. Level 3 Func D Sub T
 C 47.0 A 46.5 I 0.0
 B 44.9 V 0.0
 S. Gas Tank Monitor. Level 3 Func D Sub G 832 psi Intox 850 psi Reg
 T. Configure Wet Bath Sim. Level 1 Func S
 U. Zero Alcohol Test Level 1 Func C
610329 Sim SN — Lot 0.00 Rep Value
0.000 AC 0.000 AC 0.000 AC
 V. Low AC Test. Level 1 Func C
DR5190 Sim SN — Lot 0.021 Rep Value
0.017 AC 0.017 AC 0.018 AC
 W. Linearity Test. Level 1 Func C
DR5113 Sim SN — Lot 0.297 Rep Value
0.302 AC 0.302 AC 0.303 AC
 X. Interferent Test.
 Level 1 Func B
 Acetone DR3845 Sim SN — Lot 0.05% Conc
 Display Printout
 Methanol DR3846 Sim SN — Lot 0.10% Conc
 Display Printout
 IPA DR5141 Sim SN — Lot 0.10% Conc
 Display Printout
 Y. Ambient Test
 Display Printout
 Z. Invalid Sample Test
 Display Printout
 AA. RFI Detector Test. CMS Mode
 Display Printout
 BB. Configure Dry Gas Standard. Level 1 Func S
 CC. Stability Test at 0.080AC. Level 3 Func T Sub S
0.0786 Ave. 0.0008 Std Dev 0.96% Rel Std Dev

DD. Maintenance/Repairs

1 Sept 2009
 Date
04 Sept 2009
 Date

Marty Pagan
 Signature
Deb Kashner
 Reviewed by

Intoxilyzer Test Record and Checklist
NDAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = TOXL 8164.12.00 06/09
09/01/2009 11:43

***** Printer Test *****

12 char/line
abcdefghijkl
ABCDEFGHIJKL

16 char per line
abcdefghijklmnop
ABCDEFGHIJKLMNPO

21 chars per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMNOPQRSTU

24 characters per line
abcdefghijklmnopqrstuwx
ABCDEFGHIJKLMNOPQRSTUWX

32 characters per line
abcdefghijklmnopqrstuwx123456
ABCDEFGHIJKLMNOPQRSTUWXYZ123456

42 characters per line
abcdefghijklmnopqrstuwx1234567890abcdef
ABCDEFGHIJKLMNOPQRSTUWXYZ1234567890ABCDEF


Current Instrument Setup
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Reslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Menfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 617774
Standard Cyl #? 34
Standard Expiration? 05/28/2011
Oper No? 107501

Flow Cal. Date: 06/03/2009
Slope 710
Intercept -570705

IR Calibration Date: 06/03/2009

	3ur	9ur
0th Coef(*100):	-5365	-24455
1st Coef(*100):	268636	131046
2nd Coef(*100):	2601	1309
#20 adj(ing/1*10K):	391	565

***** Test End *****

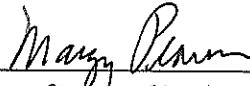

Operator Signature
MARGY PEARSON

Remarks:

Form 106-18000

80-004191
P. 3 of 17 FM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = TOXL 8164.12.00 06/09
09/01/2009 14:33

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Fail
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 107501

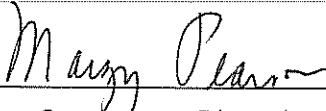

Operator Signature
MARGY PEARSON

Remarks:

DIAG

Form 106-I8000

80-004191
P. 4 of 17 file

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = TOXL 8164.12.00 06/09
09/01/2009 11:52

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No


of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 617774
Standard Cyl #? 34
Standard Expiration? 05/28/2011
Oper No? 107501

Flow Cal. Date: 06/03/2009
Slope 710
Intercept -670706

IR Calibration Date: 06/03/2009

	3um	9um
0th Coef(*100):	-5389	-24455
1st Coef(*100):	268636	131046
2nd Coef(*100):	2601	1309
H2O adj(mg/l*10k):	391	565

***** Printer Test End *****

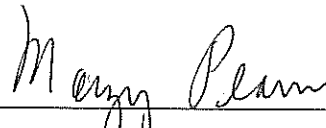

Operator Signature
MARGY PEARSON

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = TOXL 8164.12.00 06/09
09/01/2009 14:31

Test	AC	Time
01 Room Air	0.000	14:32
02 *Subject Test	REF*	14:32
03 Room Air	0.000	14:33

*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 617774
Cyl No = 34
Expiration Date = 05/28/2011
County = 08 Oper No. = 107501

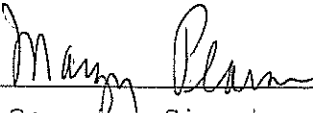

Operator Signature
MARGY PEARSON

Remarks:

Refusal

Form 106-I8000

80-004191
p. 6 of 17
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = TOXL 8164.12.00 06/09
09/01/2009 15:45

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:46
02 Std. Sol.	0.000	15:47
03 Room Air	0.000	15:47
04 Std. Sol.	0.000	15:48
05 Room Air	0.000	15:48
06 Std. Sol.	0.000	15:49
07 Room Air	0.000	15:49
08 Sim Temp = 34.0°C		

Simul Ser No = DRO000
Std Sol No = 0000
County = 08 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

Water

Form 106-I8000

80-004191
P. 7 of 17 gm


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = TOXL 8164.12.00 06/09
09/01/2009 16:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:36
02 Std. Sol.	0.017	16:37
03 Room Air	0.000	16:37
04 Std. Sol.	0.017	16:38
05 Room Air	0.000	16:39
06 Std. Sol.	0.018	16:39
07 Room Air	0.000	16:40
08 Sim Temp = 34.0°C		

Simul Ser No = DRO000
Std Sol No = 0000
County = 08 Oper No. = 107501

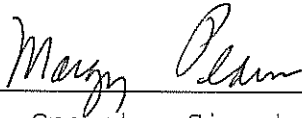

Operator Signature
MARGY PEARSON

Remarks:

LDW AC 0.021

Form 106-I8000

*80-004191
P. 8/17*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = TOXL 8164.12.00 06/09
09/01/2009 16:51

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:51
02 Std. Sol.	0.302	16:52
03 Room Air	0.000	16:53
04 Std. Sol.	0.302	16:54
05 Room Air	0.000	16:54
06 Std. Sol.	0.303	16:55
07 Room Air	0.000	16:56
08 Sim Temp = 34.0°C		

Simul Ser No = DROOOO
Std Sol No = 0000
County = 08 Oper No. = 107501

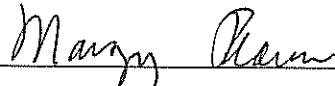

Operator Signature
MARGY PEARSON

Remarks:

Linearity

Form 106-I8000

80-004191
P. 9 of 17 FW


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = TOXL 8164.12.00 06/09
09/01/2009 17:15

Test	AC	Time
01 Room Air	0.000	17:15
02 *Subject Test	INT*	17:17
03 Room Air	0.000	17:18

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DROOOO
Std Sol No = 0000
County = 08 Oper No. = 107501

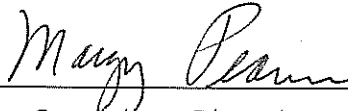

Operator Signature
MARGY PEARSON

Remarks:

Acetone

Form 106-I8000

80-004191
P. 10 of 17 *fm*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = TOXL 8164.12.00 06/09
09/01/2009 17:23

Test	AC	Time
01 Room Air	0.000	17:24
02 *Subject Test	INT*	17:25
03 Room Air	0.000	17:25

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DROOOO
Std Sol No = 0000
County = 08 Oper No. = 107501

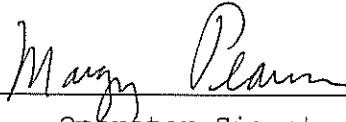

Operator Signature
MARGY PEARSON

Remarks:

MeOH

Form 106-I8000

80-004191
P. 11 of 17 AM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = TOXL 8164.12.00 06/09
09/01/2009 17:26

Test	AC	Time
01 Room Air	0.000	17:27
02 *Subject Test	INT*	17:28
03 Room Air	0.000	17:29

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DROOOO
Std Sol No = 0000
County = 08 Oper No. = 107501

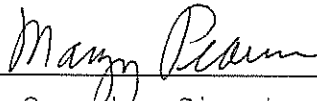

Operator Signature
MARGY PEARSON

Remarks:

1 PA

Form 106-I8000

80-004191
P.12 of 17 KA


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = TOXL 8164.12.00 06/09
09/01/2009 17:49

Test	AC	Time
01 Room Air	AMB*	17:50
02 Room Air	0.000	17:51


*Invalid Test
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DROOOO
Std Sol No = 0000
County = 08 Oper No. = 107501

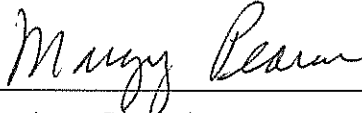

Operator Signature
MARGY PEARSON

Remarks:

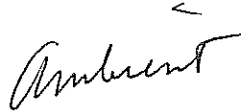

Form 106-I8000

80-004191
p. 13 of 17 *JKL*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = TOXL 8164.12.00 06/09
09/01/2009 17:51

Test	AC	Time
01 Room Air	0.000	17:52
02 Invalid Sample	X.XXX	17:52
03 Room Air	PUR*	17:53

*Purge Fail
**Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DROOOO
Std Sol No = 0000
County = 08 Oper No. = 107501

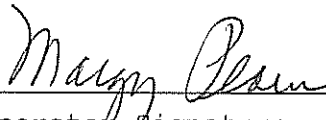

Operator Signature
MARGY PEARSON

Remarks:

Invalid Sample

Form 106-I8000

80-004191
P. 192/17 pm


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = TOXL 8164.12.00 06/09
09/01/2009 15:53

Test	AC	Time
01 Room Air	RFI*	15:54
02 Room Air	0.000	15:54

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DROOOO
Std Sol No = 0000
County = 08 Oper No. = 107501

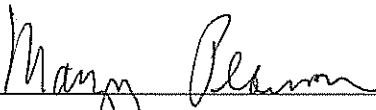

Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-004191
P. 15 of 17 PM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = TOXL 8164.12.00 06/09
09/01/2009 11:53

Test	AC	Time
01 Room Air	0.000	11:53
02 Std. Gas	0.076	11:54
03 Room Air	0.000	11:54
04 Std. Gas	0.078	11:55
05 Room Air	0.000	11:55
06 Std. Gas	0.078	11:55
07 Room Air	0.000	11:56
08 Std. Gas	0.078	11:56
09 Room Air	0.000	11:57
10 Std. Gas	0.079	11:57
11 Room Air	0.000	11:58
12 Std. Gas	0.079	11:58
13 Room Air	0.000	11:59
14 Std. Gas	0.078	11:59
15 Room Air	0.000	12:00
16 Std. Gas	0.079	12:00
17 Room Air	0.000	12:01
18 Std. Gas	0.079	12:01
19 Room Air	0.000	12:02
20 Std. Gas	0.079	12:02
21 Room Air	0.000	12:03
22 Std. Gas	0.079	12:03
23 Room Air	0.000	12:04
24 Std. Gas	0.079	12:04
25 Room Air	0.000	12:05
26 Std. Gas	0.079	12:05
27 Room Air	0.000	12:06
28 Std. Gas	0.079	12:06
29 Room Air	0.000	12:07
30 Std. Gas	0.079	12:07
31 Room Air	0.000	12:08
32 Std. Gas	0.079	12:08
33 Room Air	0.000	12:09
34 Std. Gas	0.079	12:09
35 Room Air	0.000	12:09
36 Std. Gas	0.079	12:10
37 Room Air	0.000	12:10
38 Std. Gas	0.079	12:11
39 Room Air	0.000	12:11
40 Std. Gas	0.078	12:12
41 Room Air	0.000	12:12
42 Cal Check Stats:		
43 Average	0.0786	
44 Std Dev	0.0008	
45 Rel Std Dev	0.96%	

80-204191
P. 16 of 17 FM
Stability
#1 of 2
MAP

Operator Signature
MARGY PEARSON

Remarks: LOT No 617774
Cyl No 34
Exp Date 5/28/2011
Form 106-I8000

Stability
2 of 2
MAP

80-004191
P. 12 of 12