

*✓ CML*

**INTOXILYZER 5000 G and 8000 INSPECTION**

Serial No.: 80-002666

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR5144 Lot No. — Rep AC 0.021 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR5187 Lot No. — Rep AC 0.290 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR5114 Lot No. — Rep AC 0.100 ✓
- J. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. <u>617774</u>	Cyl No. <u>48</u>	Exp. Date <u>28 May 2011</u>
Test 1 <u>0.080</u>	Test 4 <u>0.081</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.081</u>	Test 8 <u>0.080</u>
Test 3 <u>0.079</u>	Test 6 <u>0.081</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

L. Remarks/Maintenance record: Annual inspection

02 Sept 09  
Date  
4 Sept 09  
Date

Deb Kashner  
Field Inspector's Signature  
Mary Olson  
Reviewed by

*P. 1 of 9 FM*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-002666  
Location = TOXL                    8164.12.00 06/09  
09/01/2009                            16:38

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        No  
# of Print Copies?           1  
Select Std (D/W/I)?        Wet  
Standard Value?            0.110  
Standard Solution #?        PS1228  
Sim Serial #?               777777  
Oper No?                    777777

Flow Cal. Date:              09/10/2007  
Slope                        677  
Intercept                    -603196

IR Calibration Date:        09/10/2007  
                                        3um                            9um

-----  
0th Coef(\*100):            -14148                    -8312  
1st Coef(\*100):            274152                    135499  
2nd Coef(\*100):            1387                      855  
H2O adj(mg/l\*10k):        430                        237

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
Operator Signature  
DEB KASHUR

Remarks:



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-002666  
Location = TOXL      8164.12.00 06/09  
09/02/2009      10:56

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:56
02 Std. Sol.	0.019	10:57
03 Room Air	0.000	10:58
04 Std. Sol.	0.019	10:58
05 Room Air	0.000	10:59
06 Std. Sol.	0.019	11:00
07 Room Air	0.000	11:00

08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature  
DEB KASHUR

Remarks:

Low AC

Form 106-I8000

P. 3af9  
FM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-002666  
Location = TOXL      8164.12.00 06/09  
09/02/2009      13:40

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:41
02 Std. Sol.	0.289	13:42
03 Room Air	0.000	13:43
04 Std. Sol.	0.297	13:43
05 Room Air	0.000	13:44
06 Std. Sol.	0.296	13:45
07 Room Air	0.000	13:45

08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*Linearity test*

Form 106-I8000

*P. 4 of 9  
JRM*

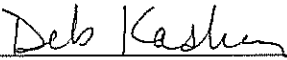
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-002666  
Location = TOXL                    8164.12.00 06/09  
09/02/2009                            15:01

Test	AC	Time
01 Room Air	0.000	15:02
02 *Subject Test	INT*	15:02
03 Room Air	0.000	15:03

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Simul Ser No = DR0000  
Std Sol No = 000  
County = 08                            Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB KASHUR

Remarks:

*Interferent check*  
Form 106-I8000

*P. Saf 9  
JFM*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-002666  
Location = TOXL                8164.12.00 06/09  
09/02/2009                      14:54

Test	AC	Time
01 Diagnostic	OK	14:55
02 Room Air	RFI*	14:55
03 Room Air	0.000	14:56

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                      Weight = 150  
Test = DUI                              Cit = NA  
Dr. Lic. = ND/DIS921456  
Simul Ser No = DR0000  
Std Sol No = 000  
County = 08                              Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

*RFI test*

Form 106-I8000

*P. Leaf 9  
SAM*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-002666  
Location = TOXL                    8164.12.00 06/09  
09/02/2009                            08:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:41
02 Std. Gas	0.080	08:41
03 Room Air	0.000	08:42
04 Std. Gas	0.080	08:42
05 Room Air	0.000	08:42
06 Std. Gas	0.079	08:43
07 Room Air	0.000	08:43

Lot No = 617774  
Cyl No = 48  
Exp Date = 05/28/2011  
County = 08                            Oper No. = 777777

Deb Kashur  
Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*  
Form 106-I8000

*P. 7 of 9*  
*JK*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-002666  
Location = TOXL      8164.12.00 06/09  
09/02/2009      08:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:44
02 Std. Gas	0.081	08:44
03 Room Air	0.000	08:45
04 Std. Gas	0.081	08:45
05 Room Air	0.000	08:46
06 Std. Gas	0.081	08:46
07 Room Air	0.000	08:47

Lot No = 617774  
Cyl No = 48  
Exp Date = 05/28/2011  
County = 08      Oper No. = 777777

*Deb Kashur*

Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*

Form 106-I8000

*P. Saf 9  
1572*



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-002666  
Location = TOXL                8164.12.00 06/09  
09/02/2009                      08:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:48
02 Std. Gas	0.080	08:48
03 Room Air	0.000	08:49
04 Std. Gas	0.080	08:49
05 Room Air	0.000	08:50
06 Std. Gas	0.080	08:50
07 Room Air	0.000	08:51

Lot No = 617774  
Cyl No = 48  
Exp Date = 05/28/2011  
County = 08                      Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*

Form 106-I8000

*P. 9/9  
JAN*