

✓ 0 mvl

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-003071

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓

- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5144 Lot No. — Rep AC 0.021
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5187 Lot No. — Rep AC 0.290
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓
Sim SN DR5114 Lot No. — Rep AC 0.100
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓

K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. <u>617774</u>	Cyl No. <u>36</u>	Exp. Date <u>28 May 2011</u>
Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.080</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>
Average <u>0.081</u>		

L. Remarks/Maintenance record: Annual inspection

02 Sept 2009
Date

Deb Kasher
Field Inspector's Signature

4 Sept 2009
Date

Mary Pearson
Reviewed by

08/09

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = TOXL 8164.12.00 06/09
09/01/2009 16:54

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:54
02 Std. Sol.	0.019	16:55
03 Room Air	0.000	16:56
04 Std. Sol.	0.019	16:57
05 Room Air	0.000	16:57
06 Std. Sol.	0.019	16:58
07 Room Air	0.000	16:59

08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Low AC

Form 106-I8000

80-003071
p. 3 of 9
JKW

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = TOXL 8164.12.00 06/09
09/02/2009 08:49

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:49
02 Std. Sol.	0.277	08:50
03 Room Air	0.000	08:51
04 Std. Sol.	0.287	08:52
05 Room Air	0.000	08:52
06 Std. Sol.	0.290	08:53
07 Room Air	0.000	08:54
08 Sim Temp = 34.0°C		

Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Linearity test

Form 106-I8000

80-003071
p. 4 of 9 Jan

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = TOXL 8164.12.00 06/09
09/02/2009 14:14

Test	AC	Time
01 Room Air	0.000	14:14
02 *Subject Test	INT*	14:15
03 Room Air	0.000	14:16

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 617774
Cyl No = 36
Expiration Date = 05/28/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent check

Form 106-I8000

80-003071
P. 549
JAN

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = TOXL 8164.12.00 06/09
09/02/2009 14:17

Test	AC	Time
01 Diagnostic	OK	14:18
02 Room Air	RFI*	14:18
03 Room Air	0.000	14:19

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 617774
Cyl No = 36
Expiration Date = 05/28/2011
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

80-003071
P. 6 of 9

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = TOXL 8164.12.00 06/09
09/02/2009 13:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:36
02 Std. Gas	0.080	13:36
03 Room Air	0.000	13:37
04 Std. Gas	0.081	13:37
05 Room Air	0.000	13:38
06 Std. Gas	0.080	13:38
07 Room Air	0.000	13:39

Lot No = 617774

Cyl No = 36

Exp Date = 05/28/2011

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003071
P. 7 of 9

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = TOXL 8164.12.00 06/09
09/02/2009 13:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:57
02 Std. Gas	0.080	13:57
03 Room Air	0.000	13:58
04 Std. Gas	0.081	13:58
05 Room Air	0.000	13:59
06 Std. Gas	0.081	13:59
07 Room Air	0.000	13:59

Lot No = 617774

Cyl No = 36

Exp Date = 05/28/2011

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-18000

80-003071
P. 8 of 9
[Signature]

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = TOXL 8164.12.00 06/09
09/02/2009 14:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:06
02 Std. Gas	0.081	14:06
03 Room Air	0.000	14:07
04 Std. Gas	0.081	14:07
05 Room Air	0.000	14:08
06 Std. Gas	0.081	14:08
07 Room Air	0.000	14:09

Lot No = 617774

Cyl No = 36

Exp Date = 05/28/2011

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003071
p. 9 of 9 *km*