

V cmr

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004186

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5140 Lot No. — Rep AC 0.021 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5113 Lot No. — Rep AC 0.297 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. — Rep AC 0.10% + Acetone ✓
- J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. 617774 Cyl No. 24 Exp. Date 5/28/2011

Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>MAP 0.080</u>	Test 9 <u>0.080</u>

Average 0.080 0.081

L. Remarks/Maintenance record: Annual Inspection

<u>2 Sept 09</u> Date	<u>Mary Plam</u> Field Inspector's Signature
<u>04 Sept 2009</u> Date	<u>Deb Kasher</u> Reviewed by

08/09

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = TOXL 8164.12.00 06/09
09/02/2009 15:13

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:14
02 Std. Sol.	0.017	15:15
03 Room Air	0.000	15:15
04 Std. Sol.	0.018	15:16
05 Room Air	0.000	15:17
06 Std. Sol.	0.017	15:17
07 Room Air	0.000	15:18

08 Sim Temp = 34.0°C

Simul Ser No = DR00000

Std Sol No = 000

County = 08

Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

Low AC 0.021

Form 106-I8000

80-004186
P. 3 of 9 PM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = TOXL 8164.12.00 06/09
09/02/2009 15:30

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:31
02 Std. Sol.	0.296	15:32
03 Room Air	0.000	15:32
04 Std. Sol.	0.296	15:33
05 Room Air	0.000	15:34
06 Std. Sol.	0.296	15:34
07 Room Air	0.000	15:35


08 Sim Temp = 34.0°C

Simul Ser No = DR00000

Std Sol No = 000

County = 08

Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

Linearity

Form 106-I8000

80-004186
p. 4 of 9 pm


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = TOXL 8164.12.00 06/09
09/02/2009 15:37

Test	AC	Time
01 Room Air	0.000	15:38
02 *Subject Test	INT*	15:38
03 Room Air	0.000	15:39

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR00000
Std Sol No = 000
County = 08 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

Acetone + Alc

Form 106-I8000

80-004186
P. Sol 9/14


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = TOXL 8164.12.00 06/09
09/02/2009 14:46

Test	AC	Time
01 Room Air	RFI*	14:47
02 Room Air	0.000	14:47

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = DR00000
Std Sol No = 000
County = 08 Oper No. = 107501

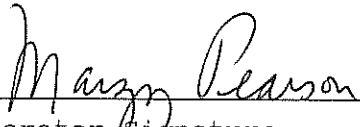

Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-004186
P. 6 of 9 File

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = TOXL 8164.12.00 06/09
09/01/2009 14:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:10
02 Std. Gas	0.080	14:11
03 Room Air	0.000	14:11
04 Std. Gas	0.080	14:11
05 Room Air	0.000	14:12
06 Std. Gas	0.080	14:12
07 Room Air	0.000	14:13

Lot No = 617774

Cyl No = 24

Exp Date = 05/28/2011

County = NA

Oper No. = NA


Operator Signature

Remarks:

ACA

Form 106-I8000

80-004186
P. 2 of 9 KTH


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003064
Location = TOXL 8164.12.00 06/09
09/01/2009 14:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:19
02 Std. Gas	0.080	14:20
03 Room Air	0.000	14:20
04 Std. Gas	0.080	14:21
05 Room Air	0.000	14:21
06 Std. Gas	0.081	14:22
07 Room Air	0.000	14:22

Lot No = 612623
Cyl No = 9
Exp Date = 05/28/2011
County = 08 Oper No. = 107501

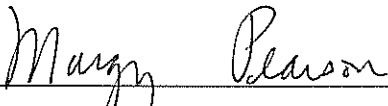

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-18000

80-004186
P. 8 of 9

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = TOXL 8164.12.00 06/09
09/02/2009 12:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:16
02 Std. Gas	0.080	12:16
03 Room Air	0.000	12:17
04 Std. Gas	0.080	12:17
05 Room Air	0.000	12:18
06 Std. Gas	0.080	12:18
07 Room Air	0.000	12:18


Lot No = 617774

Cyl No = 24

Exp Date = 05/28/2011

County = 08

Oper No. = 107501

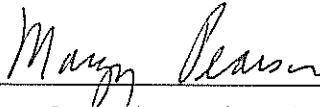

Operator Signature
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-004186
P. 9-19-11