



ETHANOL BREATH STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59282 (5-2011)

| | | |
|--|---|---------------------------------------|
| Chemical Test Operator Name (Print) Lori Flaten | | |
| Location MANDAN | Intoxilyzer® Serial Number 80-004939 | |
| Gas Lot Number 11611080AZ | Gas Cylinder Number 82 | Gas Expiration Date 06-01-2013 |

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0. 079 AC
 - B. 0. 079 AC
 - C. 0. 075 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Breath Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

| | |
|---|-----------------------|
| Chemical Test Operator Signature | Date 09-02-11 |
| Reviewed By (Crime Laboratory Use Only) Deb Kashur | Date 03 Apr 12 |

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = MAND 8164.13.00 06/09
09/02/2011 10:40

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 10:41 |
| 02 Std. Gas | 0.079 | 10:41 |
| 03 Room Air | 0.000 | 10:42 |
| 04 Std. Gas | 0.079 | 10:42 |
| 05 Room Air | 0.000 | 10:43 |
| 06 Std. Gas | 0.075 | 10:43 |
| 07 Room Air | 0.000 | 10:44 |

Lot No = 11611080A2
Cyl No = 82
Exp Date = 06/01/2013
County = 30 Oper No. = 068019



Operator Signature
LORI E. FLATEN

Remarks: