

✓ CML

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-003057

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5144 Lot No. — Rep AC 0.021
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5187 Lot No. — Rep AC 0.290
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓
Sim SN DR5114 Lot No. — Rep AC 0.100
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. 617774 Cyl No. 38 Exp. Date 28 May 2011
Test 1 0.081 Test 4 0.081 Test 7 0.080
Test 2 0.080 Test 5 0.081 Test 8 0.080
Test 3 0.080 Test 6 0.080 Test 9 0.081
Average 0.080

L. Remarks/Maintenance record: Annual inspection

03 Sept 09
Date
4 Sept 09
Date

Deb Keshner
Field Inspector's Signature
Mary Pearson
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.12.00 06/09
09/03/2009 09:02

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 585711
Standard Cyl #? 03/03/2011
Standard Expiration? 11
Oper No? 777777

Flow Cal. Date: 04/07/2008
Slope 718
Intercept -644065

IR Calibration Date: 06/05/2008

	3um	9um
0th Coef(*100):	-22884	-16302
1st Coef(*100):	265055	140885
2nd Coef(*100):	151	572
H2O adj(mg/l*10k):	508	293

***** Printer Test End *****

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Print Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.12.00 06/09
09/03/2009 11:57

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:58
02 Std. Sol.	0.017	12:00
03 Room Air	0.000	12:00
04 Std. Sol.	0.017	12:01
05 Room Air	0.000	12:01
06 Std. Sol.	0.018	12:02
07 Room Air	0.000	12:03

08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Low AC

Form 106-I8000

80-003057
P. 3 of 9
JKH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.12.00 06/09
09/03/2009 09:54

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:55
02 Std. Sol.	0.280	09:56
03 Room Air	0.000	09:56
04 Std. Sol.	0.281	09:57
05 Room Air	0.000	09:58
06 Std. Sol.	0.282	09:58
07 Room Air	0.000	09:59

08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Linearity test

Form 106-I8000

80-003057
P. 4 of 9 pm

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.12.00 06/09
09/03/2009 09:46

Test	AC	Time
01 Room Air	0.000	09:47
02 *Subject Test	INT*	09:47
03 Room Air	0.000	09:48

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent test

Form 106-I8000

80-003057
P. Saf. 9
JK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.12.00 06/09
09/03/2009 12:48

Test	AC	Time
01 Diagnostic	OK	12:48
02 Room Air	RFI*	12:48
03 Room Air	0.000	12:49

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-18000

80-003057
P. Leaf 9
SKM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.12.00 06/09
09/03/2009 09:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:35
02 Std. Gas	0.081	09:35
03 Room Air	0.000	09:35
04 Std. Gas	0.080	09:36
05 Room Air	0.000	09:36
06 Std. Gas	0.080	09:37
07 Room Air	0.000	09:37

Lot No = 617774
Cyl No = 38
Exp Date = 05/28/2011
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003057
P. 7/27/09

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.12.00 06/09
09/03/2009 09:38

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:39
02 Std. Gas	0.081	09:39
03 Room Air	0.000	09:39
04 Std. Gas	0.081	09:40
05 Room Air	0.000	09:40
06 Std. Gas	0.080	09:41
07 Room Air	0.000	09:41

Lot No = 617774
Cyl No = 38
Exp Date = 05/28/2011
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003057
P. Poffman

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.12.00 06/09
09/03/2009 09:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:42
02 Std. Gas	0.080	09:42
03 Room Air	0.000	09:43
04 Std. Gas	0.080	09:43
05 Room Air	0.000	09:44
06 Std. Gas	0.081	09:44
07 Room Air	0.000	09:45

Lot No = 617774

Cyl No = 38

Exp Date = 05/28/2011

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003057
P 9 of 9
JKH