

V cme

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-003066

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5144 Lot No. Rep AC 0.021 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5187 Lot No. Rep AC 0.290 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR5114 Lot No. Rep AC 0.100 ✓
- J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓

K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. <u>612623</u>	Cyl No. <u>39</u>	Exp. Date <u>28 May 2011</u>
Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.082</u>
Test 2 <u>0.082</u>	Test 5 <u>0.081</u>	Test 8 <u>0.082</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.082</u>
Average <u>0.081</u>		

L. Remarks/Maintenance record: ^{DK} Deb Annual inspection

04 Sept 2009
Date

Deb Kashur
Field Inspector's Signature

4 Sept 2009
Date

Mary Pluss
Reviewed by

80-003066
P. 1 of 9
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.12.00 06/09
09/04/2009 08:43

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 612623
Standard Cyl #? 39
Standard Expiration? 05/28/2011
Oper No? 777777

Flow Cal. Date: 04/08/2008
Slope 696
Intercept -643215

IR Calibration Date: 06/04/2008

	3um	9um
0th Coef(*100):	-11861	-9861
1st Coef(*100):	275638	138672
2nd Coef(*100):	165	672
H2O adj(mg/l*10k):	357	250

***** Printer Test End *****

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.12.00 06/09
09/04/2009 09:20

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:20
02 Std. Sol.	0.017	09:21
03 Room Air	0.000	09:22
04 Std. Sol.	0.018	09:22
05 Room Air	0.000	09:23
06 Std. Sol.	0.019	09:24
07 Room Air	0.000	09:24

08 Sim Temp = 34.0°C

Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Low AC

Form 106-18000

80-003066
p. 3 of 9
JKL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.12.00 06/09
09/04/2009 09:37

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:37
02 Std. Sol.	0.280	09:38
03 Room Air	0.000	09:39
04 Std. Sol.	0.281	09:40
05 Room Air	0.000	09:40
06 Std. Sol.	0.282	09:41
07 Room Air	0.000	09:41

08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Linearity test

Form 106-I8000

80-003066
P. 4 of 9
KAM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.12.00 06/09
09/04/2009 09:03

Test	AC	Time
01 Room Air	0.000	09:06
02 *Subject Test	INT*	09:06
03 Room Air	0.000	09:07

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent test

Form 106-I8000

80-003066
p. 5 of 9
SM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.12.00 06/09
09/04/2009 09:15

Test	AC	Time
01 Diagnostic	OK	09:16
02 Room Air	RFI*	09:16
03 Room Air	0.000	09:16

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

80-003066
P. 6 of 9
YR

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.12.00 06/09
09/04/2009 08:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:35
02 Std. Gas	0.082	08:35
03 Room Air	0.000	08:36
04 Std. Gas	0.082	08:36
05 Room Air	0.000	08:37
06 Std. Gas	0.082	08:37
07 Room Air	0.000	08:38

Lot No = 612623
Cyl No = 39
Exp Date = 05/28/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-18000

80-003066
P. 7 of 9
JK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.12.00 06/09
09/04/2009 08:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:25
02 Std. Gas	0.081	08:26
03 Room Air	0.000	08:26
04 Std. Gas	0.082	08:27
05 Room Air	0.000	08:27
06 Std. Gas	0.081	08:27
07 Room Air	0.000	08:28

Lot No = 612623

Cyl No = 39

Exp Date = 05/28/2011

County = 08

Oper No. = 777777

Deb Keshur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003066
P. 8 of 9
FL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.12.00 06/09
09/04/2009 08:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:31
02 Std. Gas	0.081	08:32
03 Room Air	0.000	08:32
04 Std. Gas	0.081	08:33
05 Room Air	0.000	08:33
06 Std. Gas	0.081	08:34
07 Room Air	0.000	08:34

Lot No = 612623
Cyl No = 39
Exp Date = 05/28/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003066
P. 9 of 9
RM