



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (1-2010)

Serial Number: <u>80-004943</u>	Instrument Location: <u>STAN</u>
Reason for Install/Repair: <input type="checkbox"/> Install After Lab Repair/Inspection <input checked="" type="checkbox"/> Other (Specify) <u>Moved to another location in building.</u>	

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level 1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: _____

[Signature]
(Field Inspector Signature)

09/04/10
Date

Deb Kashner
Reviewed By

09 Sept 2010
Date

80-004943
p 53 of 56
KM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = STAN 8164.13.00 06/09
09/04/2010 20:03

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%\$^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%\$^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 709352
Standard Cyl #? 34
Standard Expiration? 12/23/2011
Oper No? 100019

Flow Cal. Date: 01/18/2010
Slope 693
Intercept -665387

IR Calibration Date: 01/18/2010

	3um	9um
0th Coef(*100):	-20351	-24446
1st Coef(*100):	248542	131200
2nd Coef(*100):	2180	1190
H2O adj(mg/l*10k):	594	450

***** Printer Test End *****

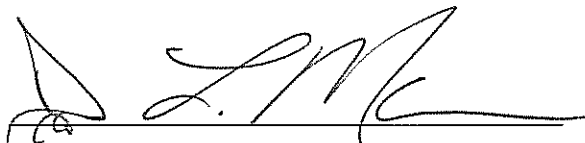

Operator Signature
DUANE L. MARMON

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = STAN 8164.13.00 06/09
09/04/2010 20:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	20:04
02 Std. Gas	0.079	20:05
03 Room Air	0.000	20:05
04 Std. Gas	0.079	20:05
05 Room Air	0.000	20:06
06 Std. Gas	0.079	20:06
07 Room Air	0.000	20:07

Lot No = 709352
Cyl No = 34
Exp Date = 12/23/2011
County = 31 Oper No. = 100019

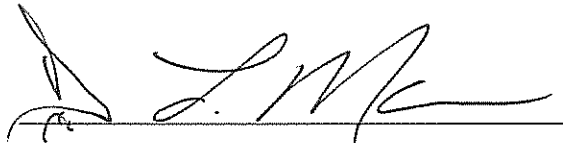

Operator Signature
DUANE L. MARMON

Remarks:

Form 106-I8000

80-004943
P. 55 of 56
YH


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = STAN 8164.13.00 06/09
09/04/2010 20:10

Test	AC	Time
01 Room Air	RFI*	20:11
02 Room Air	0.000	20:12

*Invalid Test
Inhibited - RFI

Sub Name = SUBJECT, TEST A
Sub DOB = 02/22/1990
Sub Sex = Female Weight = 350
Test = OTH Cit = 12345
Dr. Lic. = ND/SUB901234
Lot No = 709352
Cyl No = 34
Expiration Date = 12/23/2011
County = 31 Oper No. = 100019

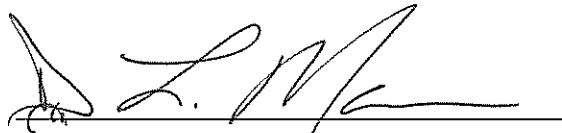

Operator Signature
DUANE L. MARMON

Remarks:

Form 106-I8000

80-004943
P. 56 of 56
LHM