

BREATH ALCOHOL EQUIPMENT
REPAIR AND MAINTENANCE

Instrument: 8000 Serial Number: 80-005946 Location: TOXL

1. Instrument repairs/maintenance performed:

Inspection after CMI repairs completed.

2. Does the instrument require further testing?
If Yes, continue.
If No, sign and date below.

Yes or No

3. If needed, set the time, date, and location.

✓

4. Does the flow need to be calibrated?
If Yes, attach paperwork.

Yes or No

5. Does the optical bench need to be calibrated?
If Yes, attach test records.

Yes or No

6. Does the gas regulator need to be calibrated?

Yes or No

PSI gauge reading 240 PSI display reading 229

Set instrument to Wet Bath and run tests:

7. Low AC. Use < 0.03 AC in ACA mode. Attach test record.

✓

Sim SN: DR3374 Lot #: 14240 AC: 0.020

8. Linearity Test. Use ≥ 0.25 AC in ACA mode. Attach test record.

✓

Sim SN: DR3841 Lot #: 201408C AC: 0.300

Set instrument to Gas and run tests:

9. Print test. Attach test record. ✓
10. Interferent Check. Use a 0.10 AC ethanol plus 0.05% acetone in ABA mode. Attach test record. ✓
- Sim SN: DR5144 Lot #: ICS 2 AC: 0.10 AC + 0.05% acetone
11. RFI Check. Run in CMS mode. Key radio during any room air or subject test. Attach test record. ✓
12. Calibration Check. Use a valid 0.080 AC Ethanol Gas Standard. Run three calibration sets in ACA mode. Attach test records. ✓

Lot No. 29614080A2 Cylinder No. 4 Expiration Date 12/05/2016

Test 1 0.080
Test 2 0.080
Test 3 0.080

Test 1 0.081
Test 2 0.080
Test 3 0.080

Test 1 0.080
Test 2 0.081
Test 3 0.080

Average 0.080

Deb Kashu
Field Inspector's Signature

Charles Elder
Reviewed by

~~28 Aug 15~~ ^{DK} 09 Sept 15
Date

09/09/2015
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = TOXL 8164.13.00 06/09
08/28/2015 08:41

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:41
02 Std. Sol.	0.019	08:42
03 Room Air	0.000	08:42
04 Std. Sol.	0.020	08:43
05 Room Air	0.000	08:44
06 Std. Sol.	0.020	08:44
07 Room Air	0.000	08:45

08 Sim Temp = 34.0°C

Simul Ser No = DR3374

Std Sol No = 14240

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

low Ac 0.020 Ac

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = TOXL 8164.13.00 06/09
08/28/2015 08:49

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:50
02 Std. Sol.	0.304	08:51
03 Room Air	0.000	08:51
04 Std. Sol.	0.303	08:52
05 Room Air	0.000	08:53
06 Std. Sol.	0.303	08:53
07 Room Air	0.000	08:54

08 Sim Temp = 34.0°C

Simul Ser No = DR3841
Std Sol No = 201408C
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = TOXL 8164.13.00 06/09
08/28/2015 09:06

Test	AC	Time
01 Diagnostic	OK	09:07
02 Room Air	RFI*	09:07
03 Room Air	0.000	09:07

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 4
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

RFI check

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = TOXL 8164.13.00 06/09
08/28/2015 09:13

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:14
02 Std. Gas	0.080	09:14
03 Room Air	0.000	09:15
04 Std. Gas	0.080	09:15
05 Room Air	0.000	09:16
06 Std. Gas	0.080	09:16
07 Room Air	0.000	09:17

Lot No = 29614080A2
Cyl No = 4
Exp Date = 12/05/2016
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

calibration check 0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = TOXL 8164.13.00 06/09
08/28/2015 09:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:19
02 Std. Gas	0.081	09:19
03 Room Air	0.000	09:19
04 Std. Gas	0.080	09:20
05 Room Air	0.000	09:20
06 Std. Gas	0.080	09:21
07 Room Air	0.000	09:21

Lot No = 29614080A2
Cyl No = 4
Exp Date = 12/05/2016
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080Ac

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = TOXL 8164.13.00 06/09
08/28/2015 09:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:22
02 Std. Gas	0.080	09:22
03 Room Air	0.000	09:23
04 Std. Gas	0.081	09:23
05 Room Air	0.000	09:24
06 Std. Gas	0.080	09:24
07 Room Air	0.000	09:25

Lot No = 29614080A2
Cyl No = 4
Exp Date = 12/05/2016
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080AC

Form 106-I8000