



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59281 (03-2014)

Serial Number <i>80-005948</i>	Instrument Location <i>WCSU / Highway Patrol Room</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *N/A*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Bern W. Huse</i> #1442	Date <i>9-12-2016</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>16 Sept 2016</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Eden* Certified Date: *18 Nov. 2016*



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005948  
Location = GRAF      8164.13.00 06/09  
09/12/2016      12:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:22
02 Std. Gas	0.079	12:22
03 Room Air	0.000	12:22
04 Std. Gas	0.079	12:23
05 Room Air	0.000	12:23
06 Std. Gas	0.078	12:24
07 Room Air	0.000	12:24

Lot No = 01316080A2  
Cyl No = 27  
Exp Date = 02/05/2018  
County = 50      Oper No. = 130890

Bradley Wingarter #1442

9-12-2016

Operator Signature  
BRADLEY WINGERTER

Remarks: Calibration check for set-up

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005948  
Location = GRAF      8164.13.00 06/09  
09/12/2016      12:37

Test	AC	Time
01 Diagnostic	OK	12:39
02 Room Air	RFI*	12:40
03 Room Air	RFI*	12:40

\*Invalid Test  
Inhibited - RFI

Sub Name = DUI, JOHN J  
Sub DOB = 11/12/1988  
Sub Sex = Male      Weight = 145  
Test = DUI      Cit = 1234567  
Dr. Lic. = ND/DUI885678  
Lot No = 01316080A2  
Cyl No = 27  
Expiration Date = 02/05/2018  
County = 50      Oper No. = 130890

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

 #1442

Operator Signature  
BRADLEY WINGERTER

9/12/16

Remarks: RFI - Testing for set-up