

✓ Database

INTOXILYZER® 5000 G AND 8000
INSTALLATION AND REPAIR CHECKOUT

AKH

Mountrail Co SO


Serial No.: 80-003061 Instrument Location: STAN

Reason for Install/Repair: _____ Install After Lab Repair/Inspection
X Other (specify) New instrument

Check When Done:

- X A. Surge Protector Installed/Properly Grounded.
- n/a B. Telephone Line Connected to Intoxilyzer® KB.
- X C. Breath Tube Heated.
- X D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- X E. Ethanol Breath Standard: _____ Connected X Installed.
(Include Form 105-G if New Cylinder Installed.)
- X F. Review/Enter Gas Setup (Level 1, Function S). (Intoxilyzer® 8000 Only.)
- X G. Print Test (Level 1, Function P). (Attach Test Record.)
- X H. ACA Test (Level 1, Function C). (Attach Test Record.)
- X I. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- J. Repair and/or Maintenance Performed: _____

09/30/09
Date
05 Oct 09
Date


Field Inspector's Signature
Deb Kashur
Reviewed By

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = STAN 8164.12.00 06/09
09/30/2009 10:58

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNQPQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNQPQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 659358
Standard Cyl #? 43
Standard Expiration? 09/03/2011
Oper No? 100019

Flow Cal. Date: 06/15/2009
Slope 737
Intercept -507305

IR Calibration Date: 06/04/2008
 3um 9um

0th Coef(*100):	-10373	-9786
1st Coef(*100):	269813	141172
2nd Coef(*100):	1454	683
H2O adj(mg/l*10k):	474	231

***** Printer Test End *****



Operator Signature
DUANE L. MARMON

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = STAN 8164.12.00 06/09
09/30/2009 11:06

Test	AC	Time
01 Diagnostic	OK	11:09
02 Room Air	RFI*	11:09
03 Room Air	RFI*	11:09

*Invalid Test
Inhibited - RFI

Sub Name = TEST, SUBJECT P
Sub DOB = 05/22/1999
Sub Sex = Female Weight = 350
Test = OTH Cit = 123456
Dr. Lic. = ND/TES992345
Lot No = 659358
Cyl No = 43
Expiration Date = 09/03/2011
County = 31 Oper No. = 100019

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

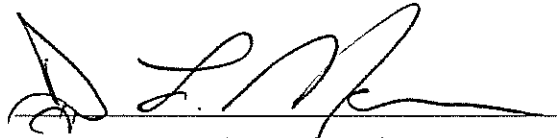

Operator Signature
DUANE L. MARMON

Remarks:

Form 106-18000

80-003061
p. 4 of 4
DM