

Database ✓

**INTOXILYZER® 5000 G AND 8000
INSTALLATION AND REPAIR CHECKOUT**

Serial No.: 80-003066 Instrument Location: 6FPD ✓

Reason for Install/Repair: _____ Install After Lab Repair/Inspection
 Other (specify) new install

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® KB.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Ethanol Breath Standard: _____ Connected Installed.
(Include Form 105-G if New Cylinder Installed.)
- F. Review/Enter Gas Setup (Level 1, Function S). (Intoxilyzer® 8000 Only.)
- G. Print Test (Level 1, Function P). (Attach Test Record.)
- H. ACA Test (Level 1, Function C). (Attach Test Record.)
- I. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- J. Repair and/or Maintenance Performed: _____

10-01-09
Date
20 Oct 09
Date

Am Yardi 049
Field Inspector's Signature
Deb Keshur
Reviewed By

80-003066
P. 1 of 4
JK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = GFPD 8164.12.00 06/09
10/01/2009 17:39

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

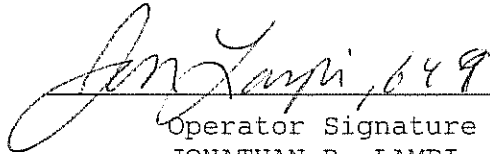
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 659358
Standard Cyl #? 12
Standard Expiration? 09/03/2011
Oper No? 039947

Flow Cal. Date: 04/08/2008
Slope 696
Intercept -643215

IR Calibration Date: 06/04/2008

	3um	9um
0th Coef(*100):	-11861	-9861
1st Coef(*100):	275638	138672
2nd Coef(*100):	165	672
H2O adj(mg/l*10k):	357	250

***** Printer Test End *****


Operator Signature
JONATHAN R. LAMPI

Remarks:

Printer test for initial install at GFPD

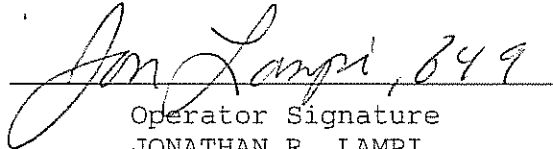
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = GFPD 8164.12.00 06/09
10/01/2009 17:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:41
02 Std. Gas	0.082	17:41
03 Room Air	0.000	17:42
04 Std. Gas	0.081	17:42
05 Room Air	0.000	17:43
06 Std. Gas	0.082	17:43
07 Room Air	0.000	17:43

Lot No = 659358
Cyl No = 12
Exp Date = 09/03/2011
County = 18 Oper No. = 039947


Operator Signature
JONATHAN R. LAMPI

Remarks:

ACA test for initial install at GFPD

Form 106-18000

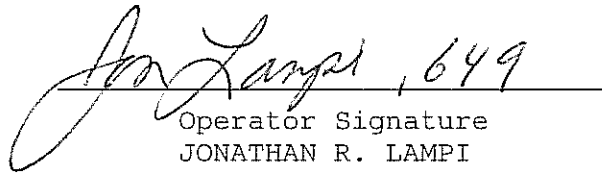
80-003066
p. 3 of 4
FLM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = GFPD 8164.12.00 06/09
10/01/2009 17:47

Test	AC	Time
01 Room Air	RFI*	17:51
02 Room Air	0.000	17:51

*Invalid Test
Inhibited - RFI


Operator Signature
JONATHAN R. LAMPI

Remarks:

RFI test for initial install at GFPD

Form 106-I8000