

Database
Chm

**INTOXILYZER® 5000 G AND 8000
INSTALLATION AND REPAIR CHECKOUT**

Serial No.: 80-003058 Instrument Location: Valley City Police Dept

Reason for Install/Repair: _____ Install After Lab Repair/Inspection
 Other (specify) new instrument

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® KB.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Ethanol Breath Standard: Connected _____ Installed.
(Include Form 105-G if New Cylinder Installed.)
- F. Review/Enter Gas Setup (Level 1, Function S). (Intoxilyzer® 8000 Only.)
- G. Print Test (Level 1, Function P). (Attach Test Record.)
- H. ACA Test (Level 1, Function C). (Attach Test Record.)
- I. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- J. Repair and/or Maintenance Performed: _____

10/02/2009 mwm
09/29/2009
Date
13 Oct 09
Date

[Signature]
Field Inspector's Signature
Deb Kashur
Reviewed By

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003058
Location = VALL 8164.12.00 06/09
10/02/2009 09:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:45
02 Std. Gas	0.081	09:46
03 Room Air	0.000	09:46
04 Std. Gas	0.081	09:47
05 Room Air	0.000	09:47
06 Std. Gas	0.081	09:47
07 Room Air	0.000	09:48

Lot No = 659358

Cyl No = 47

Exp Date = 09/03/2011

County = 02

Oper No. = 119502



Operator Signature
MARK W. MCDONALD

Remarks:

Form 106-I8000

80-003058
P. 248 PW

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003058
Location = VALL 8164.12.00 06/09
09/29/2009 16:59

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 659358
Standard Cyl #? 47
Standard Expiration? 09/03/2011
Oper No? 119502

Flow Cal. Date: 04/07/2008
Slope 698
Intercept -678970

IR Calibration Date: 06/05/2008

	3um	9um
0th Coef(*100):	-18867	-13178
1st Coef(*100):	274682	135660
2nd Coef(*100):	1139	678
H2O adj(mg/l*10k):	434	216

***** Printer Test End *****

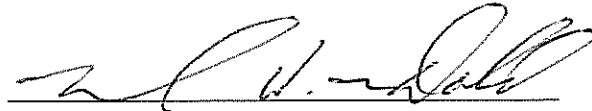

Operator Signature
MARK W. MCDONALD

Remarks:

Print Test

Form 106-I8000

80-003058
P. 3 of 8

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003058
Location = VALL 8164.12.00 06/09
09/29/2009 17:06

Test	AC	Time
01 Diagnostic	OK	17:09
02 Room Air	RFI*	17:09
03 Room Air	PUR**	17:10

*Invalid Test
 Inhibited - RFI
**Purge Fail

Sub Name = TEST, TEST TEST
Sub DOB = 01/01/1999
Sub Sex = Male Weight = 120
Test = OTH Cit = 1
Dr. Lic. = ND/99999999
Lot No = 659358
Cyl No = 47
Expiration Date = 09/03/2011
County = 02 Oper No. = 119502

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

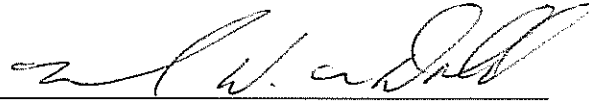

Operator Signature
MARK W. MCDONALD

Remarks:

Form 106-I8000

80-003058
P. 4 of 8

Mark -

I looked over the installs today. I need one more ACA from you. You sent in an ACA for the Cylinder Install form but another one needs to be done for the 8000 Install Checkout.

I'm sending this all back to you so it doesn't get separated & lost. Here is what you need to do:

1. Run an ACA (Level 1, Function C)
2. The date you run this ACA will be the date on the install - 8000 - form (not the cylinder install form since that date is still OK).
So put a line through your old date of 09/29/2009 write the date you ran this last needed ACA, ^{on top of old date} and also initial.
3. Send back all copies.

Thanks Mark,

Deb Kashur
328-6159

**INTOXILYZER® 5000 G AND 8000
INSTALLATION AND REPAIR CHECKOUT**

Serial No.: 80-003058 Instrument Location: Valley City Police Dept.

Reason for Install/Repair: _____ Install After Lab Repair/Inspection

Other (specify) new instrument

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® KB.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Ethanol Breath Standard: Connected _____ Installed.
(Include Form 105-G if New Cylinder Installed.)
- F. Review/Enter Gas Setup (Level 1, Function S). (Intoxilyzer® 8000 Only.)
- G. Print Test (Level 1, Function P). (Attach Test Record.)
- H. ACA Test (Level 1, Function C). (Attach Test Record.)
- I. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- J. Repair and/or Maintenance Performed: _____

09/29/2009
Date


Field Inspector's Signature

Date

Reviewed By

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003058
Location = VALL 8164.12.00 06/09
09/29/2009 17:06

Test	AC	Time
01 Diagnostic	OK	17:09
02 Room Air	RFI*	17:09
03 Room Air	PUR**	17:10

*Invalid Test
 Inhibited - RFI
**Purge Fail

Sub Name = TEST, TEST TEST
Sub DOB = 01/01/1999
Sub Sex = Male Weight = 120
Test = OTH Cit = 1
Dr. Lic. = ND/99999999
Lot No = 659358
Cyl No = 47
Expiration Date = 09/03/2011
County = 02 Oper No. = 119502

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

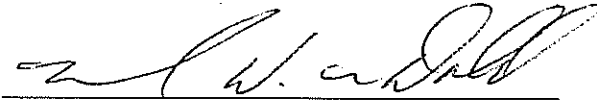

Operator Signature
MARK W. MCDONALD

Remarks:

Form 106-I8000

80-003058
P. 8 of 8